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Contents

1	For your safety	21	4	Menu	30
1.1	General safety notes	21	4.1	Activation	30
1.2	Definitions of alert icons	21	4.2	Operation	30
2	Description	22	4.3	Symbols on the display	30
2.1	Product overview	22	4.3.1	Menu	30
2.2	Intended use	23	4.3.2	Function	30
2.3	Device symbols and displays	23	4.4	Menu items	31
2.3.1	Device symbols	23	5	Fault	31
2.3.2	Keys	23	6	Technical Manual	31
3	Use	24	7	Troubleshooting	32
3.1	Prerequisites	24	8	Maintenance	33
3.1.1	Requirements of the test subject	24	8.1	Maintenance intervals	33
3.2	Preparations for use	24	8.2	Cleaning	33
3.2.1	Inserting or replacing the batteries	24	9	Storage	33
3.2.2	Charging rechargeable NiMH batteries (optional)	25	10	Disposal	33
3.2.3	Fitting the wrist strap	25	10.1	Disposal of the device	33
3.2.4	Fitting the wrist strap (optional)	25	10.2	Battery disposal	33
3.2.5	Replacing the mouthpiece receptacle	26	11	Technical data	34
3.2.6	Attach the mouthpiece	26	12	Order list	36
3.2.7	Switching the device on and off	27			
3.3	Automatic measurements	27			
3.3.1	Carry out another measurement	28			
3.3.2	Errors when giving the breath sample	29			
3.4	Additional sampling options	29			
3.4.1	Passive sampling	29			
3.4.2	Manual sampling	30			

1 For your safety

1.1 General safety notes

- Before using this product, carefully read the Instructions for Use.
- Strictly follow the Instructions for Use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the Intended use section of this document.
- Do not dispose of the Instructions for Use. Ensure that they are retained and appropriately used by the product user.
- Only trained and competent users are permitted to use this product.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommends a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Use only genuine Dräger spare parts and accessories, or the proper functioning of the product may be impaired.
- Do not use a faulty or incomplete product and do not modify the product.
- Notify Dräger in the event of any component fault or failure.
- This device is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.
- Electrical connections to devices which are not listed in these Instructions for Use should only be made following consultation with the respective manufacturers or an expert.

1.2 Definitions of alert icons

The following alert icons are used in this document to provide and highlight areas of the associated text that require a greater awareness by the user. A definition of the meaning of each icon is as follows:

CAUTION

Indicates a potentially hazardous situation which, if not avoided, could result in physical injury, or damage to the product or environment. It may also be used to alert against unsafe practices.


NOTICE

Indicates additional information on how to use the product.


2 Description

2.1 Product overview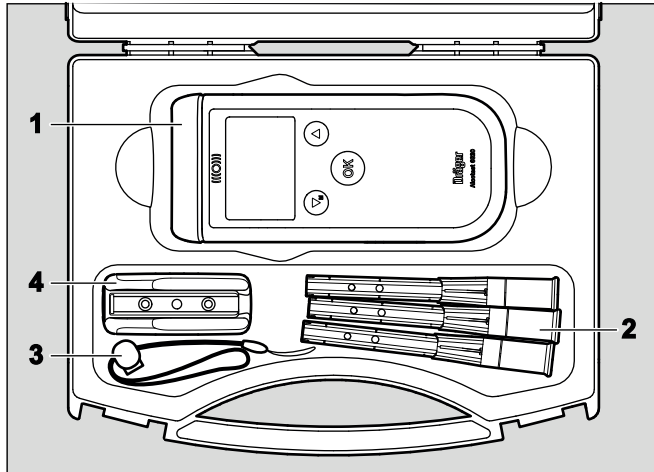

Case

- 1 Measuring device
Dräger Alcotest 6820
- 2 Mouthpieces, 3 pieces
- 3 Wrist strap
- 4 Mouthpiece receptacle

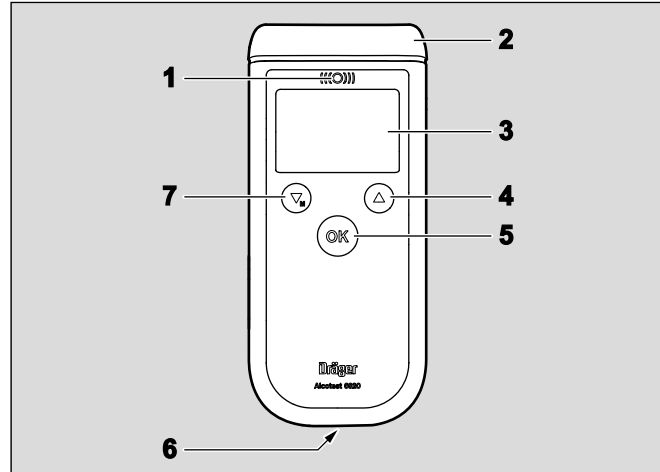Measuring device (incl. 2 batteries)

- 1 Light red/green/yellow - optical interface
- 2 Mouthpiece receptacle
- 3 Display
- 4 Up key
- 5 OK key
- 6 Charging and communication socket
- 7 Down/Menu key

2.2 Intended use

The Dräger Alcotest® 6820¹⁾ is used to quickly measure the alcohol concentration in a person's breath. The device is suitable for portable and battery-powered operation.

2.3 Device symbols and displays

The measured breath alcohol value is specified in the basic device setting, in the unit of mg/L (= milligrams of ethanol per litre of breath). The device can be converted to all common units of measurement.

2.3.1 Device symbols

In these Instructions for Use a number of symbols are used in accordance with the message on the display of the device.

Battery symbol

Indicates the batteries' state of charge:

Battery full

Battery 2/3 full

Battery 1/3 full

Battery empty

NOTICE

The symbols on the name plate and on the dust guard of the charging and communication socket have the following meanings: Attention, read the Instructions for Use! Please refer to the Instructions for Use before changing the battery.

2.3.2 Keys

OK key
Confirms entries or selected functions.
Key for switching the device on and off.

UP and DOWN key
Setting values and selection of menu functions.

1) Alcotest® is a registered trademark of Dräger.

3 Use

3.1 Prerequisites

- The ambient air must be free from alcohol and solvent vapours (disinfectants).
- Keep sufficient distance to antennas of mobile phones and transmitter sites.

3.1.1 Requirements of the test subject

- Observe a waiting period of at least 15 minutes after the last oral alcohol intake!
Residual alcohol in the mouth can distort the measurement. Even aromatic beverages (e. g. fruit juice), alcoholic mouth sprays, medical juices and drops and, belching and vomiting may result in false data. Also observe a waiting period of at least 15 minutes in these cases. Rinsing the mouth with water or non-alcoholic drinks does not replace the waiting time!
- The test person should breathe regularly and calmly before the sample is taken. Avoid breathing in and out quickly through the mouth.
- The test person must be capable of exhaling the minimum breath volume of 1.2 l (works setting). For the test, the breath flow must fall within a certain minimum exhalation time (works setting 2 seconds). The volume and the time can be changed via the menu, if required.

3.2 Preparations for use

3.2.1 Inserting or replacing the batteries

CAUTION

Explosion hazard!

Do not throw used batteries/rechargeable batteries into a fire or open by force. Do not recharge alkaline batteries. Batteries/rechargeable batteries must be disposed of in accordance with local and national regulations.

Only use the charger indicated in the order list.

Only charge rechargeable NiMH batteries.

NOTICE

To ensure that the device can be used again immediately after the alkaline batteries or the rechargeable NiMH batteries are discharged, it is recommended to have replacement batteries available.

Replace the batteries within 5 minutes, otherwise the save time will be reset and must be set again.

1. Release the screw plug on the battery lid (e. g. using a coin) and remove the battery lid.
2. Insert the batteries and make sure that the polarity is correct.
3. Close the battery lid and tighten the screw plug.

Required batteries:

- 2 x 1.5 V alkaline type (AAA, LR6, AA) or,
- 2 rechargeable batteries - 1.2 V NiMH (AAA, LR6, AA)


3.2.2 Charging rechargeable NiMH batteries (optional)

Rechargeable NiMH batteries can be charged via a special power source in the device.


- Connect the charger (see section 12 on page 36) to the charging/communication socket.
During the charging process that battery display pulses from empty to full.

3.2.3 Fitting the wrist strap

1. Undo the screw plug on the battery lid (e. g. using a coin) and remove the battery lid.
2. Insert the eye supplied in the opening and fix with the screws (2x) supplied.
3. Guide the fastening lug of wrist strap through the eye.
4. Pull the lower part of the wrist strap through the strap.
5. Close the battery lid and tighten the screw plug.
6. Pull the cord adjuster over the wrist strap.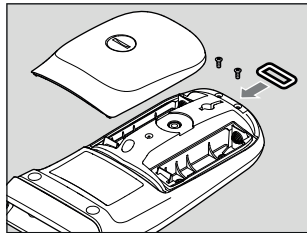


00333400.eps

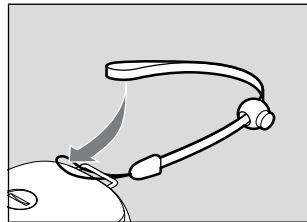00433400.eps

3.2.4 Fitting the wrist strap (optional)

1. Undo the screws of the mouthpiece receptacle and remove the mouthpiece receptacle.
2. Undo the screw plug on the battery lid (e. g. using a coin) and remove the battery lid.
3. Insert the upper eye of the wrist strap in the upper opening on the device.
4. Place the mouthpiece receptacle on the device and fix with the screws (2x).
5. Insert the bottom lug of the wrist strap in the bottom opening on the device and fix with the screws (2x) of the wrist strap.
6. Close the battery lid and tighten the screw plug.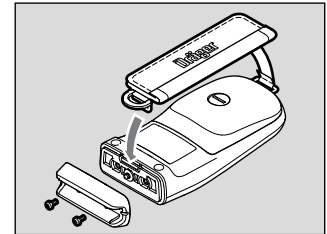

00533400.eps


3.2.5 Replacing the mouthpiece receptacle

NOTICE

Replacing the mouthpiece receptacle is not permitted in all countries. Please observe national regulations!

1. Undo the screws on the mouthpiece receptacle.
2. Remove the mouthpiece receptacle.
3. Insert new mouthpiece receptacle on the device, observe the desired mouthpiece orientation.
4. Fix the mouthpiece receptacle with screws (2x).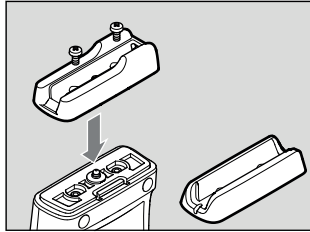


00633400.eps

3.2.6 Attach the mouthpiece


1. Remove the mouthpiece from the packaging. For reasons of hygiene, leave the protective wrapping around the mouth contact area until the mouthpiece is firmly attached to the device.

- With mouthpiece version A:
 - With the fluted markings (= press-on surface) facing upwards, hold the mouthpiece at an angle, push it upwards and press it into the mouthpiece receptacle. The mouthpiece audibly snaps into place. The mouthpiece can be used facing to the right or to the left.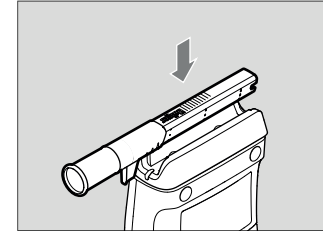

00733400.eps

- With mouthpiece version B:
 - With the fluted markings (= press-on surface) facing upwards, hold the mouthpiece at an angle in the mouthpiece receptacle and press it into the mouthpiece receptacle. The mouthpiece audibly snaps into place.

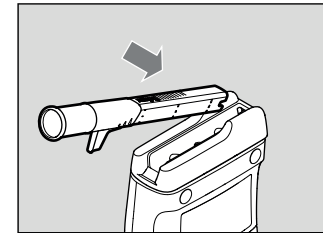00833400.eps

2. Remove all packaging.

3.2.7 Switching the device on and off

To switch the device on:

- Press and hold the OK key for approx. 1 second until the start window appears. A self check is carried out. Device information is displayed.

01033400.eps

If the next service date has expired:

- Have the device calibrated, see section 7 on page 32.

To switch the device off:

- Press and hold the OK key. The device will switch off after approx. 2 seconds.

01133400_en.eps

NOTICE

The device will switch off automatically approx. 4 minutes after the last use. When the batteries or rechargeable batteries are discharged, the device switches off automatically – the battery symbol flashes.

3.3 Automatic measurements

CAUTION

Do not allow any high alcohol concentrations on the sensor, e. g., by rinsing the mouth with a high proof alcohol shortly before the measurement, as this will shorten the service life of the sensor.


NOTICE

A new mouthpiece must be inserted for each test subject.

With automatic measurement, the device takes the sample automatically as soon as the test subject has given a breath sample that is sufficient into the mouthpiece of the device and the end of the breath sample has been recognised by the device.

To carry out an automatic measurement:

1. Insert a new mouthpiece into the holder (see section 3.2.6 on page 26).
2. Switch on the device (see section 3.2.7 on page 27). After approx. 2 seconds, **READY** will appear on the display and a short acoustic signal will sound. At the same time the current test number is displayed. The light flashes green. The device is ready to use.
3. Ask the test subject to blow into the mouthpiece evenly and without interruption, as long as possible.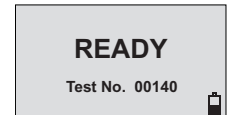


01233400_en.eps


While the breath sample is given, a continuous tone sounds, the green light is illuminated and, **Breath sample** appears on the display.

When the breath sample has been given, the green light goes out, the continuous tone will stop and a short clicking sound can be heard.

On the display, **Analysis** appears.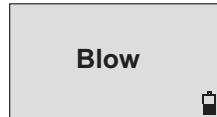

01433400_en.eps

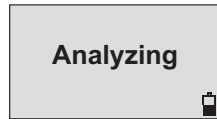
01433400_en.eps

The test result will appear on the display after 5 to 25 seconds (depending on the the device temperature and the measured concentration).

01533400.eps

4. Push the mouthpiece upwards out of the mouthpiece receptacle.
5. Dispose of the mouthpiece in accordance with local regulations.

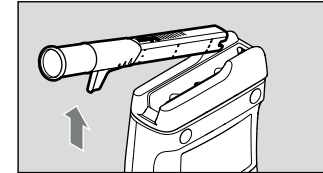00933400.eps

3.3.1 Carry out another measurement

- Pressing the OK key will prepare a new measurement. During the recovery time of the sensors, see section 11 on page 34, the number of seconds and a rotating bar appear on the display.

01633400.eps

NOTICE

Periodically switching off the device does not reduce this waiting period!


NOTICE

The measurement result is displayed with the corresponding unit. The unit has been set to mg/L (milligrams of ethanol per litre of breath) at the plant and can be changed via the menu. In addition to the result of the measurement, an acoustic signal is output, for instance:
PASS – 1 x, ALERT – 2 x, FAIL – 3 x

While the result is shown on the display, a measurement protocol can be generated with the optional Dräger Mobile Printer. Completed tests can be re-printed from the **Last Tests** menu.

3.3.2 Errors when giving the breath sample

If the volume of the breath sample is too small, **Volume too small** will appear on the display.

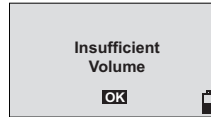01733400_en.eps

A short acoustic signal sounds and the light flashes red.

- To repeat, press the OK key.

After approx. 2 seconds, the device is ready for measuring.

In the case of very irregular exhalation, e. g. sudden termination or suction at the end of the breath sample, **Invalid sample** appears on the display

01833400_en.eps

A short acoustic signal sounds and the light flashes red.

- To repeat the measurement, press the OK-Taste. Time until it is ready for another measurement: 4 to 30 seconds.

3.4 Additional sampling options

The functions can be switched on or off in the menu (see section 4 on page 30), also in combination.


3.4.1 Passive sampling

With passive sampling, the sampling is initiated manually by the device user. Possible applications include the testing of ambient air or the breath of uncooperative (e. g. unconscious) persons for the presence of alcohol.

To carry out passive sampling:

1. Switch on the device (see section 3.2.7 on page 27).
2. Hold the device which is ready for measuring into the air to be analysed, **without a mouthpiece**.
3. Briefly press the OK key.

Display 1: No alcohol present

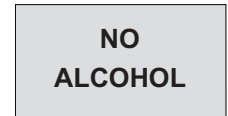01933400_en.eps

Display 2: Alcohol is present

02033400_en.eps

4. Press the OK key for a new measurement.

3.4.2 Manual sampling

Sampling can also be triggered manually if the test subject cannot produce the required minimum breath volume.

1. Preparation as for automatic measurement (see section 3.3 on page 27).
2. When "Breath sample" appears on the display after exhalation has begun, briefly press the OK key.

When the result is displayed, the message **Manual** also appears on the display.

02133400_en.eps

4 Menu

4.1 Activation

Switch on the device. When **READY** appears or, when the rotating bar is displayed, the menu functions can be accessed.

- For menu: briefly press the key.

4.2 Operation

- Select the functions using and .
- The selected function is displayed inverted.
- Activation of the selected function with the OK key.
- Selecting **Back to measurement** brings the device back into the measurement mode when the OK key is pressed.

NOTICE

If no key is pressed within 120 seconds, the device automatically returns to the measurement mode.

4.3 Symbols on the display

4.3.1 Menu

- Return to the next menu level up
- Menu item with submenu
- Open menu item
- Available function
- Further entries below the visible entries
- Further entries above the visible entries

4.3.2 Function

- Activated function
- 1/6 Current display (1) of the total display number (6)

4.4 Menu items


Last tests

The device displays the last 5000 tests. A smaller number can be set in the range 0 - 249.

Selecting the Tests with \blacktriangledown and \blacktriangle .

Go back with the OK key.

The data sets can be transferred to a PC using the "Diagnostics" PC software.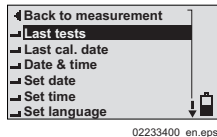

02233400_en.eps

Last cal. date

Displays the date of the last calibration and the last accuracy test. Go back with the Hokey.

Date & time

Displays the current date and the time. Go back with the OK key.

Date (optional)

Displays and sets the date and the date format. Change with \blacktriangledown and \blacktriangle ; confirm with the OK key.

Time (optional)

Displays and sets the time, the time format and the automatic changeover from summer to winter time. Change with \blacktriangledown and \blacktriangle ; confirm with the OK key.

Select language (optional)

Selects the display language.

Set contrast

Change with \blacktriangledown and \blacktriangle . Go back with the OK key.


Check accuracy (optional)

Additional test of the measuring accuracy within the calibration period

5 Fault

An error message (refer to page 32) will appear if a device fault occurs.

The number that appears below the message is used for service functions. If the fault persists even after the device has been switched on and off several times and/or the battery has been removed (for a minimum of 2 seconds) and re-inserted, contact Dräger.

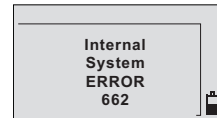02333400_en.eps

6 Technical Manual

Further information, e. g. on calibration or the selection of other measurement units are contained in the Technical Manual.

7 Troubleshooting

Fault	Cause	Remedy
Device switches off automatically.	Device has been ready to measure for longer than 4 minutes.	Switch the device back on.
	(Rechargeable) batteries drained.	Replace see section 3.2.1 on page 24 or charge batteries.
Menu is exited automatically in measuring mode.	Device automatically jumps to the measuring function after 120 seconds.	Select menu item again.
Device cannot be switched on.	Batteries discharged.	Insert new batteries or charge batteries.
Volume too low / sample invalid.	Test subject is blowing intermittently or not hard enough.	Have test subject blow with more force and without interruption.
Service date expired.	The service interval has expired.	Have safety checks carried out by experts, e. g. by Dräger.
No text on the display.	Device fault, potentially caused by electrostatic discharge.	Remove the batteries (minimum of 2 seconds) and re-insert, use DrägerService, if required.

Error message on the display	Cause	Remedy
"Calibration Test incorrect"	Calibration test not successful.	Repeat calibration test, if required, by experts, e. g. by Dräger.
↑ ↑ ↑	Measured value exceeds the display area.	Use lower concentrations.
"Internal System ERROR" with error number	Device fault, measurement not possible.	Remove the batteries (minimum of 2 seconds) and re-insert, use DrägerService, if required.
"Service Date expired"	The service interval has expired.	Have safety checks carried out by experts, e. g. by Dräger.
"no battery charge"	No rechargeable NiMH batteries in the device.	Remove the batteries. Insert rechargeable NiMH batteries and charge.

8 Maintenance

8.1 Maintenance intervals

As necessary:

- Replacing the batteries, see section 3.2.1 on page 24.

Every six months:

- Check calibration.

8.2 Cleaning

CAUTION

Damage to the device! Do not immerse the device in liquids and make sure liquids do not penetrate the connectors.

For cleaning:

- Wipe with a disposable cloth.
- Do not use detergents that contain alcohol or solvents.

9 Storage

- Do not store the device in environments which contain alcohol.
- The device consumes power, also when switched off. Therefore, remove the battery for extended disuse.

CAUTION

For long term storage:

Remove the batteries if the device is not used for 6 months and longer. This will prevent damage due to leaking battery acid.

10 Disposal

Dispose of the product in accordance with the applicable rules and regulations.

10.1 Disposal of the device

According to the 2002/96/EC directive, the product must not be disposed of as municipal waste. It is therefore labelled with the symbol depicted here.

The product can be returned to Dräger free of charge. For information, please contact the national sales organisations and Dräger.

10.2 Battery disposal

According to the 2002/96/EC directive, batteries and rechargeable batteries must not be disposed of as municipal waste. They are, therefore, marked with the adjoining symbol. Collect and dispose of batteries and rechargeable batteries at battery collection centres, in accordance with applicable regulations.

11 Technical data

Measurement principle	Electrochemical sensor
Measuring range	
Breath alcohol concentration	0.00 to 2.50 mg/L (Ethanol mass per breath volume at 34 °C and 1013 hPa.)
Blood alcohol concentration	0.00 to 5.00‰ (Ethanol mass per blood volume and/or mass at 20 °C and 1013 hPa. For the above data, a conversion factor of 2100 is used for breath and blood alcohol concentration data (works setting, country-specific conversion factors can be configured).
Indication range	
	The display is identical to the measuring range.
	If the measurement result is out of range, the display will show three arrows "↑↑↑" indicating that the measuring range has been exceeded.

Ambient conditions	
During operation	-5 to +50 °C 10 to 100 % r.h. (non-condensing) 600 to 1400 hPa
During storage	-40 to +70 °C
Taking samples	
Breath volume	1,2 l (pre-set)
Breath flow	>10 l/min (pre-set)
Duration of breath	2 s (pre-set)
Measurement accuracy	
Standard deviation of the repeatability with ethyl alcohol standard	
up to 0.50 mg/L	0,008 mg/l
≥ 0.50 mg/L	1.7 % of the measured value
up to 1.00 ‰	0,017 ‰ or
≥ 1,00 ‰	1.7% of the measured value whichever value is higher
Sensitivity drift	typically 0.6% of the measured value/month

Waiting time until device is ready to use

1. Measurement approx. 2 s

after a previous measurement with:
(breath alcohol concentration is indicated)

0.25 mg/L 20 s

0.50 mg/L 40 s

1.00 mg/L 80 s

> 1.5 mg/L 120 s

after a previous measurement with:
(blood alcohol concentration is indicated)

0.50 ‰ 20 s

1.00 ‰ 40 s

2.00 ‰ 80 s

> 3.00 ‰ 120 s

Values for other units of measurement can be calculated accordingly (see section 11 on page 34).

Calibration every 6 months

Dimensions 65 mm x 147 mm x 39 mm (W x H x D)

Weight approx. 260 g

Power supply

Batteries 2 x 1.5 V alkaline type
(AAA, LR6, AA)

approx. 1500 measurements or,

rechargeable batteries 2 x 1.2 V NiMH
(AAA, LR6, AA)

Interface USB,
for data transfer to PC

Protection class IP 54

CE mark Electromagnetic Compatibility
Directive (89/336/EEC)

12 Order list

Name and description	Order no.
Dräger Alcotest 6820(Standard) (Measuring device, 3 mouthpieces, batteries, wrist strap and case)	83 22 660
Dräger Alcotest 6820 (Customer-specific configuration) (Measuring device, 3 mouthpieces, batteries, wrist strap and case)	83 22 655
Accessories	
Dräger Mobile Printer	83 19 310
Case for Dräger Alcotest 6820	83 17 920
Leather case for Dräger Alcotest 6XXX (black) (belt attachment)	83 17 911
Leather case for Dräger Alcotest 6XXX (yellow) (Device can be operated from the case), with compartment for 4 mouthpieces, no belt attachment	83 17 931
Diagnostics	83 19 342
PC program for data communication with Dräger Alcotest 6820	
PC communication cable	83 19 715
Printer communication cable with mini-USB (for Dräger Mobile Printer)	83 18 657

Name and description	Order no.
Charger (power source) for charging the rechargeable NiMH batteries in the Dräger Alcotest 6810	83 16 991
Technical Manual	90 23 833
Spare parts	
Mouthpieces without non-return valve (100 pieces)	68 10 690
Mouthpieces without non-return valve (250 pieces)	68 10 825
Mouthpieces without non-return valve (1000 pieces)	68 10 830
Mouthpieces with non-return valve (100 pieces)	68 11 055
Mouthpieces with non-return valve (250 pieces)	68 11 060
Mouthpieces with non-return valve (1000 pieces)	68 11 065
Alkaline battery (1 piece)	13 35 804
NiMH battery (1 piece)	83 17 912
Paper (5 rolls), shelf life 7 years (for Dräger Mobile Printer)	83 19 002

90 33 400 - GA 4754.580 de/en
© Dräger Safety AG & Co. KGaA
Edition 01 - November 2012
Subject to alteration

Dräger Safety AG & Co. KGaA
Revalstraße 1
23560 Lübeck, Germany
Tel +49 451 882-0
Fax +49 451 882-20 80
www.draeger.com