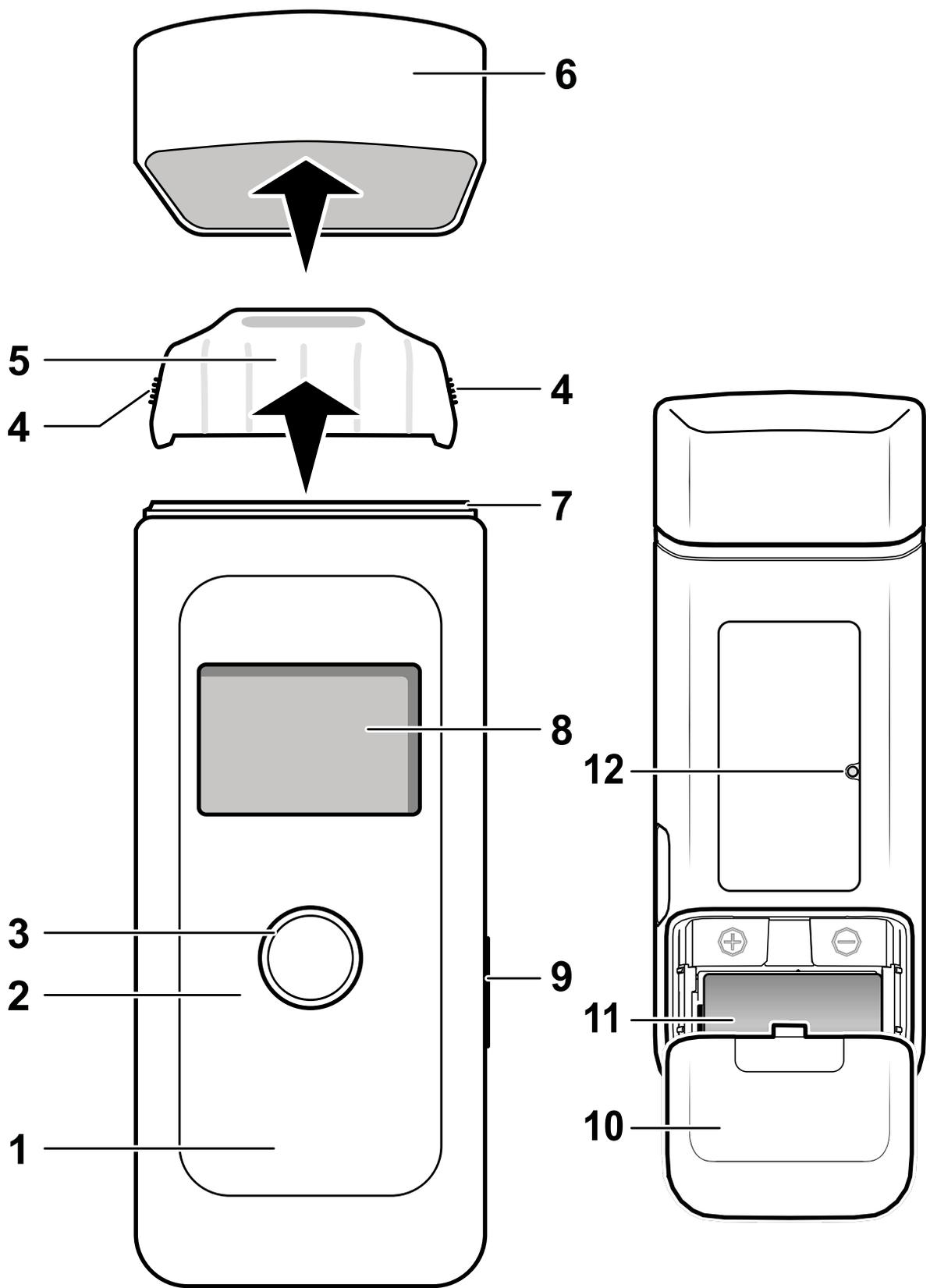
| | | | | | | | | | |
|-----------|----|-----------|-----|-----------|-----|-----------|-----|-------------|-----|
| de | 3 | nl | 87 | lt | 171 | bs | 255 | tr | 339 |
| en | 15 | da | 99 | pl | 183 | cs | 267 | ar | 351 |
| fr | 27 | fi | 111 | ru | 195 | bg | 279 | zh | 363 |
| es | 39 | no | 123 | hr | 207 | uk | 291 | zhTW | 374 |
| ca | 51 | sv | 135 | sl | 219 | ro | 303 | ja | 385 |
| pt | 63 | et | 147 | sk | 231 | hu | 315 | ko | 397 |
| it | 75 | lv | 159 | sr | 243 | el | 327 | | |

Dräger Alcotest[®] 4000 Instructions for use

00100211.ai

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1 Safety-related information

- Before using this product, carefully read these instructions for use and those of the associated products.
- Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.
- Do not dispose of the instructions for use. Ensure that they are retained and appropriately applied by the user.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommend a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Do not use a faulty or incomplete product. Do not modify the product.
- Notify Dräger in the event of any component fault or failure.
- Use only genuine Dräger spare parts and accessories. Otherwise, the proper functioning of the product may be impaired.

These instructions for use can be electronically downloaded in other languages on the corresponding product page (www.draeger.com/ifu) or can be obtained in print from Dräger or a specialist retailer free of charge.

Only use chargers, power packs, batteries or rechargeable batteries that Dräger has approved for use with this product.

2 Conventions used in this document

2.1 Definition of alert icons

The following alert icons are used in this document to inform the user of possible danger. A definition of the meaning of each alert icon is as follows:

WARNING

Indicates a potentially hazardous situation. If not avoided, it could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation. If not avoided, it could result in physical injury. It may also be used to alert against unsafe practices.

NOTICE

Indicates a potentially hazardous situation. If not avoided, it could result in damage to the product or environment.

2.2 Typographical conventions

Text

Texts written in bold mark labels on the instrument and screen texts.

In warning information, this triangle marks ways of avoiding the danger.

This symbol marks information which makes it easier to use the product.

3 Description

3.1 Product overview (see Page 2)

- 1 Alcohol measuring instrument
- 2 LED ring
- 3 Button
- 4 Gripping area on mouthpiece
- 5 Mouthpiece
- 6 Protective cap
- 7 Mouthpiece receptacle
- 8 Display
- 9 USB port (not for charging the instrument, only for service)
- 10 Battery compartment cover
- 11 Battery
- 12 Loudspeaker

3.2 Intended use

The Dräger Alcotest[®] 4000¹⁾ alcohol measuring instrument is used to measure the concentration of alcohol in human breath. The measurement is used to determine the intoxication of people. No therapeutic measures may be introduced on the basis of the measurement.

4 Use

WARNING

Small parts of the instrument can come off and be swallowed.

- ▶ The instrument is not intended for children under the age of 3.

4.1 Prerequisites for use

The instrument is supplied having been calibrated in the factory. If the calibration date is more than 12 months ago, then the instrument should be calibrated, e.g. by DrägerService.

- After the last alcohol consumption, please wait for approx. 15 minutes.
- The wait time cannot be shortened by rinsing the mouth with water or a non-alcoholic drink.
- Residual mouth alcohol or aromatic drinks (e.g. fruit juice), alcoholic mouth sprays and medicines can falsify the measurement.

1) Alcotest[®] is a registered trademark of Dräger.

- Burping and vomiting can falsify the measurement result.

4.1.1 Prior to sampling

- The test person should breathe regularly and calmly before sampling.
- The test person must be capable of providing the minimum breath volume required for sampling.

4.1.2 Preparation for use

1. Make sure that the batteries are charged and inserted.
2. Make sure that a clean mouthpiece has been fitted.

4.1.3 Switching instrument on and off

Switching on the instrument:

1. Press the button and hold for approx. 1 second.
2. Wait until it shows it is ready to make a measurement.

Switch off the instrument:

1. Press the button and hold for approx. 3 seconds.

The instrument will switch itself off 2 minutes after last being operated.

Switching off the instrument does not shorten the wait time for being ready to measure.

4.2 Initial start-up

4.2.1 Standard wizard

When the instrument is switched on for the first time, the setup assistant is opened. This step is omitted for preconfigured instruments.

The following settings are made using the setup assistant:

- Language
- Date format
- Date
- Time format
- Time
- Measurement unit

All settings can be retrospectively changed using the menu.

4.2.2 Country wizard

When the instrument is switched on for the first time, a country list opens. After selecting the required country, the country-specific settings are pre-configured. Selecting language or measurement unit manually, for example, is then no longer necessary.

All settings can be retrospectively changed using the menu.

NOTICE

Loss of default settings

If changing the battery takes more than 5 minutes, then the date and time need to be reset. It is only possible to make a measurement with the time set.

4.3 Taking a breath sample

Use a new mouthpiece for every test person.

Do not apply a high concentration of alcohol to the sensor, e.g. by rinsing the mouth with high proof alcohol shortly before making the measurement. This reduces the life span of the sensor.

1. Place new mouthpiece (order no. 83 25 250) on the mouthpiece receptacle (see page 2).
2. Press and hold the button for approx. 1 second until the startup screen appears.
3. Approx. 4 seconds later **READY** is displayed. The instrument is ready to perform a measurement.
4. Blow evenly and without interruption into the mouthpiece.
When doing so, do not block the outlet opening of the mouthpiece with a hand.
While doing so, a continuous tone will sound.

After a few seconds (depending on the instrument temperature and on the concentration measured), the measurement result is displayed.

Performing another measurement

- To prepare for a new measurement, briefly press the button. After the recovery time of the sensor (see “Technical Data” on Page 24), **READY** will be displayed.

4.4 Removing mouthpiece

1. Pull the mouthpiece up and off the mouthpiece receptacle.
2. Dispose of the mouthpiece in compliance with local regulations.

4.5 Menu control

Display the menu:

1. Briefly press the button. The menu display is shown. If the button is not pressed for 3 seconds, it returns to being ready to measure.
2. Briefly press the button again (within 3 seconds). The menu is displayed.

The menu is left automatically after 30 seconds without pressing the button.

Navigating in the menu:

- Briefly press the button. Once the last entry is reached, then the next time the button is pressed, it jumps to the top again.

Confirm the selected feature or input:

- Hold the button pressed.

Leave the menu:

- Select the **Back** menu item in sub-menus.
- Select the **Measuring** menu item in the main menu.

Change values:

1. Briefly press the button until the required setting is reached.

Press and hold the button to confirm the setting.

4.6 Menu items

The menu contains the following items:

4.6.1 Measurement results

Shows all saved results with measurement unit, date, and time, as well as the sample number and the information whether the sample was taken manually or passively.

4.6.2 Bluetooth® (optional)

 The instrument is prepared for connection to an app by Bluetooth®.

Switches the Bluetooth® feature ON or OFF respectively.

- Confirm Bluetooth® ON or OFF with the button.

 An active Bluetooth® connection is displayed as a corresponding symbol in the display.

 Bluetooth® must not be activated in an aeroplane.

4.6.3 Next service

Shows the remaining time for calibration and service in days.

4.6.4 Date & Time

Sets the date and time.

- Change and confirm the digits with the button.

4.6.5 Contrast

Adjusts the contrast.

- Select and confirm the contrast setting with the button.

4.6.6 Loudspeaker

Adjusts the volume.

- Set and confirm the volume with the button.

4.6.7 Language

Sets the language.

- Select and confirm the language with the button.

4.6.8 Software

Shows the software version number and the associated part number.

5 Troubleshooting

If the instrument malfunctions, an error message will appear. The number that appears under the message is for service purposes. If the malfunction does not disappear after switching the instrument on and off several times, then please contact DrägerService or the distributor.

| Malfunction/Error | Cause | Remedy |
|--|---|---|
| Instrument will not switch on. | Battery flat. | Change battery. |
| Volume too small / sample invalid. | Test person is not blowing strongly enough or with interruptions. | Blow with more force and without interruption into the mouthpiece. |
| Instrument outside the temperature range | Instrument is too warm or too cold. | Allow instrument to cool down or warm up. |
| ↑ ↑ ↑ (over range) | Measuring range has been exceeded. Residual alcohol in the mouth can distort the measurement. | At least 15 minutes wait time must have passed since alcohol was last consumed through the mouth. (See Page 19). To repeat, press button. |
| Volume too small | Breath sample volume too small. | Repeat breath sample. |
| Volume too large | Measuring range exceeded. | Repeat breath sample. |
| Sample disruption | Irregular breathing out, abrupt ending or breathing in | Repeat breath sample. |
| Error number | Instrument error | Switch the instrument off and on again. If the error remains, contact DrägerService or the distributor. |

6 Maintenance

Have instrument calibrated once a year as the measurement uncertainty increases over time due to drift effects. Only have calibration and maintenance carried out by authorised persons, e.g. DrägerService.

6.1 Changing battery

1. Switch off instrument.
2. Push the battery cover off.
3. Replace the battery (battery type: 3 V CR123A; order no.: 83 21 390). Ensure that the polarity is correct when doing so.
4. Push the battery cover back on.
5. Check whether the time has been set correctly.

If changing the battery takes more than 5 minutes, then the time needs to be reset. It is only possible to make a measurement with the time set.

6.2 Cleaning

The instrument should be cleaned with a cleaning agent based on ethanol.

Cleaning the instrument:

1. Wipe the instrument using a disposable cloth and the cleaning agent. Do not apply the cleaning agent directly onto the instrument. Make sure that no liquid enters the gas inlet.
2. After cleaning, leave the instrument unpacked at room temperature (20-25 °C) for at least 24 hours.
3. Then use a breath alcohol test without alcohol to ensure that there is no residual ethanol inside the instrument and that the result "0.00" is displayed.

7 Disposal

This product must not be disposed of as household waste. It is therefore marked with the adjacent symbol. You can return this product to Dräger free of charge. For information please contact the national marketing organizations or Dräger.

Batteries must not be disposed of as household waste. They are therefore marked with the adjacent symbol. Dispose of batteries and rechargeable batteries at battery collection centres as specified by the applicable regulations.

8 Technical data

| | |
|----------------------------|------------------------|
| Measuring principle | Electrochemical sensor |
|----------------------------|------------------------|

Measuring range

| | |
|------------------------------|---|
| Breath alcohol concentration | 0.00 to 2.50 mg/L (Ethanol mass per respiratory volume at 34 °C and 1013 hPa.) |
|------------------------------|---|

| | |
|-----------------------------|--|
| Blood alcohol concentration | 0.00 to 5.00 ‰ (Ethanol mass per blood volume or mass at 20 °C and 1013 hPa.) |
|-----------------------------|--|

When providing this data, a conversion factor of 2100 between the breath and blood alcohol concentration data is used (factory setting, country-specific conversion factors can be configured).

Measurement units

mg/L, µg/100 mL, µg%, µg/L, g/210 L, g/L ‰, ‰, mg/100 mL, mg%, mg/mL (country-specific configuration)

The instrument meets the requirements of EN 15964. It can be configured according to the standard.

Measured value display with configuration according to EN 15964:

| Measured value | Display |
|-----------------------|----------------|
| 0.0 mg/L to 0.03 mg/L | 0.0 mg/L |
| <0.03 mg/L | Measured value |

Ambient conditions

| | |
|------------------|---|
| During operation | -5 to +50 °C (23 °F to +122 °F) 10 to 100 % RH (non-condensing) 600 to 1300 hPa |
| During storage | -20 to +60 °C (-4 °F to +140 °F) 15 to 75 % RH (10 to 100 % RH <48 h) |

Sampling

| | |
|--------------------|-----------------------------|
| Respiratory volume | >1.2 L (default setting) |
| Breath flow | >10 L/min (default setting) |
| Breath length | >1.6 s (default setting) |

Measurement accuracy

Standard deviation of repeatability with ethanol standard

| | |
|-----------------|---|
| Up to 0.50 mg/L | 0.008 mg/L |
| ≥0.50 mg/L | 1.7 % of the measured value |
| Up to 1.00 ‰ | 0.017 ‰ or |
| ≥1.00 ‰ | 1.7 % of the measured value depending on which value is higher |

| | |
|----------------------|---|
| Drift of sensitivity | Typically 0.4 % of the measured value per month |
|----------------------|---|

Wait time for readiness to take measurement after a previous measurement of:

| | | | |
|------------|-------|---------|-------|
| 0.25 mg/L | 20 s | 0.50 ‰ | 20 s |
| 0.50 mg/L | 40 s | 1.00 ‰ | 40 s |
| 1.00 mg/L | 80 s | 2.00 ‰ | 80 s |
| >1.50 mg/L | 120 s | >3.00 ‰ | 120 s |

Calibration Recommendation: every 12 months

Dimensions 50 mm x 133 mm x 29 mm
(W x H x D)

Weight approx. 130 g

Power supply 1 x 3 V CR123A

The batteries are designed for up to 1500 measurements (depending on use and ambient conditions).

Protection class IP 52

Interface USB socket for service only

CE markings Electromagnetic compatibility

Configuration possible, see name plate