
Dräger Alcotest[®] 7000 med

gültig ab Softwareversion 1.0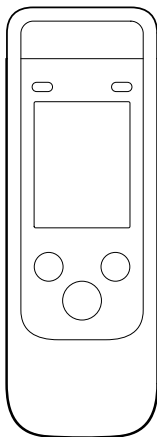

Instructions for use

de · enUS

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Contents

1	Safety information	36
2	Conventions in this document	37
2.1	Meaning of the warning notes	37
2.2	Typographical conventions	38
3	Description	38
3.1	Product overview	38
3.2	Intended use	38
3.3	Label symbol explanations	40
3.4	Display symbol explanations	41
4	Use	41
4.1	Starting the device for the first time	41
4.2	Menu	42
4.3	Prerequisites for use	42
4.4	Preparations for use	44
4.5	Breath sample	44
4.6	Functional test/quality assurance	49
4.7	Turning off the device	50
4.8	Removing the mouthpiece	50
4.9	Menu items	50
4.10	Printing results	53
4.11	Exporting data	54
5	Troubleshooting	54
6	Maintenance	56
6.1	Charging the rechargeable battery	56
6.2	Cleaning	57
7	Storage	58

8	Disposal	58
9	Technical data	58
10	Order list	61
11	Annex	61
11.1	Glossary	61

1 Safety information

- Before using this product, carefully read these instructions for use and those of the associated products.
- Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.
- Do not dispose of the instructions for use. Ensure that they are stored and used appropriately by the user.
- Only trained and competent users are permitted to use this product.
- Do not use a faulty or incomplete product. Do not modify the product.
- Notify Dräger in the event of any component fault or failure.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommend a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Use only genuine Dräger spare parts and accessories. Otherwise, the proper functioning of the product may be impaired.
- Only use the accessories listed in this document, as other accessories may result in increased electromagnetic interference or reduced electromagnetic immunity.
- Portable RF communication devices (radios), including their accessories, such as antenna cables and external antennas, should not be used closer than 30 cm (12 inches) from the parts and ducts of the device designated by the manufacturer. Failure to observe this information may result in a decline in device performance. This does not include smartphones.

- The instructions for use correspond with DIN 60601-1-2, section 5.2.1.1 f).

 These instructions for use are available in further languages and can be downloaded in electronic form on the according product website (www.draeger.com) or a printed version can be ordered free of charge via Dräger or specialist shops.

Use only chargers, power packs or batteries approved by Dräger for this product.

2 Conventions in this document

2.1 Meaning of the warning notes

The following warning notes are used in this document to notify users of possible dangers. The meanings of the warning notes are defined as follows:

Alert icon	Signal word	Classification of warning
	WARNING	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION	Indicates a potentially hazardous situation which, if not avoided, could result in injury. It may also be used to alert against unsafe practices.
	NOTICE	Indicates a potentially hazardous situation which, if not avoided, could result in damage to the product or environment.

2.2 Typographical conventions

Text	Text marked in bold denotes labeling on the device and on-screen messages.
▶	This triangle labels possible methods of avoiding the hazards mentioned in warning notices.
>	The greater-than sign denotes the navigational path in a menu.
	This symbol denotes information that make using the product easier.

3 Description

3.1 Product overview

See Figure 1 on the fold-out page:

1	Breath alcohol testing device
2	USB-C port
3	Wrist strap
4	 button (set value and carry out functional test)
5	Display
6	Mouthpiece
7	Brightness sensor
8	LEDs
9	 button (access menu, set value)
10	 button (turn device on/off, confirm entries)
11	Mouthpiece receptacle

3.2 Intended use

The Dräger Alcotest[®] breath alcohol testing device¹⁾ 7000 med is used to quickly measure the breath alcohol concentration in a person's breath in healthcare facilities and other medical settings.

User group

The Dräger Alcotest® breath alcohol testing device is used by authorized testers to carry out breath samples on patients. Authorized testers are employees who work in the medical sector and carry out breath samples as part of medical use (e.g. ambulance service, doctors, nurses). Users also include service technicians from Dräger or service technicians authorized by Dräger. The patient is not the user of the product. The application is not intended for children and young people under 5 years of age or 40 kg. There are no other restrictions in relation to gender, age, education or other sociodemographic aspects.

Clinical benefits

The device allows for quick measurement of the breath alcohol concentration, meaning that further medical treatment decisions can be made depending on the result.

Indication

Measurement of alcohol in the breathing air of a patient in the medical sector.

Contraindication

Residual mouth alcohol. Residual mouth alcohol will mean that the breath alcohol concentration (BrAC) measured via blowing will not match the blood alcohol concentration (BAC). The alcohol consumed has not yet fully dispersed in the blood. The residual mouth alcohol usually needs 15 minutes to wear off.

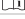
WARNING

Fire hazard!

- Do not use the device in the vicinity of flammable gases or anesthetics.
-

1) Alcotest® is a registered trademark of Dräger.

3.3 Label symbol explanations

	Medical product
	Order number
	Serial number
	Observe the instructions for use
	Disposal symbol according to WEEE Directive
	Date of manufacture
	Manufacturer
	Upper and lower temperature limits The storage temperature is stated on the packaging. The usage temperature is stated on the device.
data matrix code	UDI=Unique Device Identification (1) Device number (11) Date of manufacture (17) Expiry date (21) Serial number (93) Part number
	Single use
	The entire mouthpiece is an applied part as per type BF.
IP 54	IP=International Protection (degree of protection) (5) Protected against damaging amounts of dust, complete protection against contact (4) Protected against spray water on all sides
CE 0483	CE marking for the notified body

3.4 Display symbol explanations

	Device with mouthpiece
	Bluetooth®
	GPS
	Battery
	Printer

4 Use

4.1 Starting the device for the first time

4.1.1 Mounting the wrist strap

Use the wrist strap (3) to secure the device to the wrist (see fold-out page, Figure A).

1. Thread the fastening lug of the wrist strap through the hole in the casing and form a loop.
2. Pull the upper part of the wrist strap through the eye that has just been made.

4.1.2 Charging the rechargeable battery

Observe the following information: "Charging the rechargeable battery", page 56.

4.1.3 Turning on the device/setup wizard

 The brightness of the display automatically adapts to ambient conditions.

If the brightness sensor (7) (see Figure A on the fold-out page) is covered, the device dims the brightness of the display, making it difficult to read the display in bright light conditions.

Never cover the brightness sensor with labels etc.

- Hold down the  button for approximately 1 second to turn the device on.

⇒ The setup wizard opens.

This step does not apply in the case of preconfigured devices.

The setup wizard will configure the following settings:

- Language
- Date format
- Date
- Time format
- Time
- Measuring unit

All settings can be altered at any time via the menu.

4.2 Menu

To access the menu:

- Press the  button.

To access the context menu:

- Press the  button.

To navigate in the menu or to configure the settings:

1. Press the  or  button.
2. Press the  button to confirm the selected feature/input.

To exit the menu:

- In the main menu, select the menu entry **Back**.

4.3 Prerequisites for use

CAUTION

Risk of swallowing

Do not distort the mouthpiece in the mouth.

- The user must observe the subject during use.
-

 The device is adjusted in the factory before delivery. Dräger recommends that the device be adjusted every 12 months by DrägerService.

- The ambient air must be free of alcohol and solvent vapors (disinfectants).
- At least 15 minutes must have elapsed since the subject last consumed alcohol.
- The wait time cannot be reduced by rinsing your mouth with water or with non-alcoholic beverages.
- Residual mouth alcohol or flavored drinks (e.g. fruit juice), breath sprays containing alcohol, medicinal juices and drops can distort the result. Observe a wait time of at least 15 minutes in these cases as well.
- Belching and vomiting can distort the result.
- The device detects exhaled samples with a volume greater than 0.3 L. The subject must be able to exhale at least this volume. If the volume of the exhaled sample is greater than 0.3 L and less than 1.2 L, the message “Low volume” appears on the display in addition to the measurement result.
The displayed breath alcohol result only corresponds to the blood alcohol concentration for samples with a volume greater than 1.2 L. All breath alcohol tests with the “Low volume” message present may display a lower result than the actual blood alcohol concentration.
- For emergency use or other critical measurements, always have a second functioning device or an alternative means of measurement available.
- The seal sticker over the name plate on the reverse of the device must not be damaged.

4.4 Preparations for use

1. Ensure that the rechargeable battery is charged.
2. Turn on the device by pressing the  button.
 - ⇒ An air blank is carried out.
An air blank is a test to see whether alcohol can be detected in the ambient air, which could distort test results.
3. If no alcohol is detected, wait until the device displays that it is ready to measure (see fold-out page, Figure B: 1).

4.4.1 Before sampling

- The subject must be breathing calmly and steadily.
- The subject should be able to provide the minimum volume of breath (>1.2 L) required for sampling.

4.5 Breath sample

Overview of the different measurement modes:

	Device status	Trigger	Result
Automatic breath sample	with mouth-piece	blow until end of the breath	measured value
Low-volume breath sample	with mouth-piece	permanently active, blow until end of the breath	measured value Notice: "Low volume"
Passive breath sample	without mouth-piece	Press the  button.	alcohol or no alcohol Notice: "Passive"

4.5.1 Taking an automatic breath sample

 Use a new mouthpiece for each measurement. 

Do not allow any high alcohol concentrations on the sensor, e.g., by purging the mouth with a high proof alcohol shortly before the measurement. Doing so shortens the life span of the sensor.

 When removing the mouthpiece (see "Order list", page 61) from the packaging, ensure that the part that goes into the mouth remains in the packaging. The mouth contact area must remain protected for hygiene reasons.

 The mouthpiece can be inserted into the mouthpiece receptacle from both sides (right or left).

1. Open the mouthpiece packaging using the prefabricated perforation and remove the part of the packaging that will be placed on the device. Leave the mouth contact area protected in the packaging for hygiene reasons (see fold-out page, Figure A).
 2. Insert the mouthpiece upwards and tilted at the corner of the mouthpiece receptacle casing so that the contact surface (serrated marking) is pointing upwards (see fold-out page, Figure A, A).
 3. Hold the mouthpiece in the tilted position and slide upwards until the mounting spacer stops the movement at the bottom of the mouthpiece (see fold-out page Figure A, left-hand side).
 4. Press the mouthpiece into the mouthpiece receptacle of the device on the contact surface (see fold-out page, Figure A, B).
⇒ The mouthpiece audibly clicks into place.
 5. Remove the rest of the mouthpiece packaging.
-

 If the device displays **READY**, a measurement or follow-up measurement can be carried out.

6. Blow evenly and without interruption into the mouthpiece.
- ⇒ A circular progress bar indicating progress as a percentage value is displayed (see fold-out page, Figure B: 2).

4.5.1.1 Reading the result

 The device detects exhaled samples with a volume greater than 0.3 L. The subject must be able to exhale at least this volume. If the volume of the exhaled sample is greater than 0.3 L and less than 1.2 L, the message "Low volume" appears on the display in addition to the measurement result. The displayed breath alcohol result only corresponds to the blood alcohol concentration for samples with a volume greater than 1.2 L. All breath alcohol tests with the "Low volume" message present may display a lower result than the actual blood alcohol concentration.

 The pre-configured alcohol threshold values determine when the red indication illuminates.

In this measurement mode, the measured value is displayed as a numeric value with unit (e.g. mg/L, ‰) and test number.

The following results are possible:

- No alcohol detected (see the fold-out page, Figure B: 4). In addition, the green indication lights up. Press the  button to continue with the next measurement.

Or

- Alcohol detected (see the fold-out page, Figure B: 5). In addition, the red indication lights up. Press the  button to continue with the next measurement.

Once the sensor's recovery time has elapsed (see "Technical data", page 58), the display indicates that the device is ready for measurement.

4.5.2 Low-volume breath sample

 If a very low volume of breath is provided, the analyzed breathing air is a mixture of lung air and air from the upper respiratory tract. The measured concentration will therefore be lower than the blood alcohol concentration.

The displayed measured value does not have to match the actual blood alcohol concentration.

 This measurement mode is permanently active.

During a measurement, the device checks whether the required minimum volume has been provided and takes a breath sample.

If the subject cannot provide the required minimum volume of breath, a low-volume sample will be taken. This means that the device can display a result even if the subject has not blown enough.

1. Prepare the device in the same way as for an automatic measurement (see "Taking an automatic breath sample", page 45).
2. Briefly press the  button.

4.5.2.1 Reading the result

In this measurement mode, the measured value is displayed as a numeric value with unit. A notice is also displayed. The notice **Low Volume** indicates that the breath sample was taken in this measurement mode.

The following results are possible:

- No alcohol detected (see the fold-out page, Figure B: 6). In addition, the green indication lights up. Press the  button to continue with the next measurement.

Or

- Alcohol detected (see the fold-out page, Figure B: 7). In addition, the red indication lights up. Press the  button to continue with the next measurement.

4.5.3 Taking a passive breath sample

False-positive results

Alcohol residues that are still present, for example from a disinfection, may distort the result.

 When measuring without a mouthpiece the breathing air may also mix with the ambient air, as a result of which the breathing air will be diluted and therefore the measured concentration will be less than the concentration in the breathing air.

In the case of the passive breath sample, sampling is manually activated by the user of the device. This means that use is possible even with subjects who are not very cooperative, are not able to blow into the mouthpiece when asked or are not able to provide a sufficient breath sample.

To take a passive breath sample:

1. Hold the ready device without the mouthpiece in the air to be analyzed.
2. To trigger the measurement, briefly press the  button and do not hold it down.
3. Wait until the analysis is completed.

4.5.3.1 Reading the result

In this measurement mode the result is displayed as text. A notice is also displayed. The notice **PASSIVE** indicates that the breath sample was taken in this measurement mode.

The following results are possible:

- No alcohol detected (see the fold-out page, Figure B: 8). The display **NO ALC.** appears. In addition, the green indication lights up. Press the  button to continue with the next measurement.

Or

- Alcohol detected (see the fold-out page, Figure B: 9). The display **ALCOHOL** appears. In addition, the red indication lights up. Press the  button to continue with the next measurement.

4.6 Functional test/quality assurance

The functional test is carried out by automatically testing the ambient air every time the device is started up. Dräger recommends that you carry out an additional functional test using ethanol once a week for quality assurance purposes. Quality testing must be carried out in accordance with Section 9 MPBetreibV (German Medical Devices Operator Ordinance).

Carry out a functional test as follows:

1. Prepare a bottle with a solution containing approximately 1 g ethanol per 1 liter water or a roughly 1% ethanol flushing solution (see "Order list", page 61).
2. Remove the mouthpiece from the device.
3. Call up the context menu using the  button.
4. Choose the functional test with the  or  button and confirm with the  button.
5. Place the metal piece of the mouthpiece receptacle of the device that is ready for measurement without a mouthpiece on the opening of a bottle. Alternatively, add a few drops of the solution to an absorbent paper towel and hold the device over it — again, do not attach a mouthpiece.
6. Briefly press the  button. The device performs a measurement and displays the result.

Reading the results

If **ALCOHOL** appears on the display and the red LED illuminates:

- The sensitivity of the device to alcohol has been confirmed. The functional test with ethanol was successful.
Briefly press the  button.
The device returns to the context menu.

⇒ The device is ready for use.

If **NO ALC.** appears on the display and the green LED illuminates:

- Repeat the functional test.

If **NO ALC.** appears on the display again:

- Have the device adjusted by specialists, e.g. DrägerService.

4.7 Turning off the device

- Hold down the  button for about 3 seconds.

 The device will switch off automatically approx. 4 minutes after the last use.

4.8 Removing the mouthpiece

- Push the mouthpiece on the mounting spacer out of the mouthpiece receptacle in an upward motion and dispose of it (see fold-out page, Figure A).

4.9 Menu items

The menu (see "Menu", page 42) contains the following items:

4.9.1 Results

Displays all stored results with the measuring mode, measuring unit, date, time and sample number, including the mode in which the sample was taken.

4.9.2 Statistics

Displays how many breath samples have already been taken.

4.9.3 About screen

Displays the part number and serial number of the device.

4.9.4 Next service

Displays, among other things, the time in days until readjustment or service is due.

4.9.5 Bluetooth®

NOTICE

Risk of other devices malfunctioning!

Electrostatic discharge and electromagnetic fields can cause other devices to malfunction.

► Do not select Bluetooth® when on an aircraft.

 The device is ready to print via Bluetooth® and to connect to an app via Bluetooth®.

An active Bluetooth® connection is indicated by the Bluetooth® symbol on the display.

Turns the Bluetooth® feature on or off, displays the linked and last used printers, searches for available printers and removes printers. The relevant authorizations are also saved here.

Bluetooth on/off:

- Choose Bluetooth® on or off using the  or  button and confirm with the .

Recent devices:

- Choose the desired printer with the  or  button and confirm with the .
- ⇒ The Bluetooth® connection to the printer is established and the printer appears in the menu.

Scanning devices:

1. Press the .
- ⇒ Available printers are searched for and displayed.

 To couple and connect a printer for the first time, see the Dräger Mobile Printer BT instructions for use.

Removing devices:

1. Choose the desired printer with the  or  button and confirm with the .
 2. Confirm the security query with the .
- ⇒ The printer is removed.

4.9.6 Date and time

Sets the formats, date and time.

- Set the date or time using the  or  button and confirm with the .
- Choose the format for the date or time with the  button or  button and confirm with the .
- Change numbers with the  or  button and confirm with the .

4.9.7 Brightness

Adjusts the brightness of the display screen.

- Set the brightness with the  or  button and confirm with the .

4.9.8 Sounds

Turns the sounds on or off.

- Turn sounds on or off with the  or  button and confirm with the .

4.9.9 Language

Sets the language.

- Choose the language with the  or  button and confirm with the .

4.9.10 Automatic turning off

Sets the time in seconds until automatic shut-down.

- Set the time using the  button or  button.
- Press the  button to confirm.

4.9.11 Software version

Displays the part number for the software package and the associated version part number.

4.9.12 Sign in

For service personnel only.

- Enter the four-digit code to open the service menu.

4.9.13 Turn off

Turns off the device.

- Briefly press the  button.

⇒ The device turns off.

4.10 Printing results

The device can be connected to the Dräger Mobile Printer BT (see "Order list", page 61) via Bluetooth® to print results.

 Follow the Dräger Mobile Printer BT instructions for use.

Alternatively, the results can also be printed using the Dräger Mobile Printer.

 Follow the Dräger Mobile Printer instructions for use.

4.11 Exporting data

The data stored by the device can be exported and then analyzed in Microsoft® Excel, for example.

1. Connect the USB-C cable to the USB-C port of the device and the USB port of a PC or laptop.
⇒ The **Alcotest 7000** drive appears on the PC/laptop.
 2. Select the **Datalog** directory on the **Alcotest 7000** drive.
 3. Select the **Datalog** file in the **Datalog** directory.
 4. Select **Copy**.
 5. Select the desired drive and directory for the file export.
 6. Select **Save**.
- ✓ The file is saved.

5 Troubleshooting

An error message is displayed in the event of a device malfunction. The number below the message is for service purposes. If the error persists even after turning the device off and on multiple times, contact authorized personnel, such as DrägerService.

Serious incidents related to the product must be reported to the manufacturer and the responsible authorities.

Error/Message	Cause	Remedy
Device does not turn on.	The rechargeable batteries are empty.	Recharge the rechargeable batteries.
Insufficient volume of breath/ sample incomplete	Subject not blowing strongly enough or blowing interrupted.	Blow into the mouthpiece more strongly and without interruption. If necessary, carry out a passive breath sample.

Error/Message	Cause	Remedy
Device out of temperature range.	Device is too warm or too cold.	Allow the device to cool down or warm up.
↑ ↑ ↑ (over measuring range)	Measuring range exceeded. Residual mouth alcohol can distort the measured values.	A wait time of at least 15 minutes after alcohol was last consumed must have elapsed. Press the  button to repeat.
Insufficient volume of breath. [Only for breath samples using the mouthpiece]	Volume of breath sample insufficient.	Repeat the breath sample after pressing the  button. If necessary, carry out a passive breath sample.
Volume of breath too large. [Only for breath samples using the mouthpiece]	Measuring range exceeded.	Repeat the breath sample after pressing the  button.
Sample disruption	Irregular exhalation, abrupt cessation or inhalation.	Repeat the breath sample after pressing the  button. If necessary, carry out a passive breath sample.
Error number	Device error.	Turn the device off and then back on. If the error persists, contact DrägerService or your dealer.

Error/Message	Cause	Remedy
Adjustment interval expired	The adjustment interval has expired.	Have adjustment carried out by authorized personnel or Dräger-Service. This does not impair the device being ready to measure.

6 Maintenance

The device is delivered with default adjustment settings. The date of the last adjustment can be recalled in the menu.

Dräger recommends that the device be adjusted once a year. Have the device adjusted only by authorized persons, e.g. DrägerService.

 Comply with the national guidelines governing adjustment.

6.1 Charging the rechargeable battery

WARNING

Risk of burns and electric shock

Using the device with a power supply unit that has not been approved by Dräger may cause burns and electric shocks to the operator/subject when charging the rechargeable battery.

- ▶ Use only the power supply unit specified in the order list (see "Order list", page 61).
- ▶ Do not connect the device to the cable and do not charge the device when carrying out measurements on the subject.

⚠ WARNING**Fire and explosion hazard**

If the device is exposed to sunlight or high ambient temperatures, the battery may overheat.

- ▶ Protect the device from sunlight and high ambient temperatures.
- ▶ If the device has been exposed to sunlight or high ambient temperatures, leave it to cool down.

i Measurements cannot be performed during the charging process. Do not charge the rechargeable battery at temperatures outside the permissible temperature range of 0 to 45 °C.

i Ensure that the power supply unit is easily accessible during charging.

- Connect the USB-C cable to the USB-C port of the device and a USB charger.

⇒ The rechargeable battery will charge.

6.2 Cleaning


Clean the device using an alcohol-based cleaning agent.

To clean the device:

1. Turn off the device.
2. If necessary, disconnect the device from the power supply.
3. Remove the mouthpiece from the device.

i Ensure that no liquid gets into the metal piece of the mouthpiece receptacle on the top of the device.

4. Wipe the device down with a disposable cloth with the cleaning agent. Do not apply the cleaning agent directly to the device.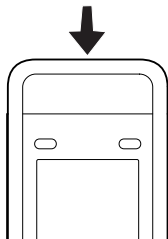

5. Do not put the device in a closed box or carton after cleaning so that the cleaning agent can evaporate. Store the device at room temperature (20-25 °C).
6. Carry out a functional test to ensure that there is no ethanol clinging to the device, which could distort future measurements. If **ALCOHOL** is displayed, do not use the device and repeat the test a while later. Carry out another functional test to ensure that there is no ethanol clinging to the device and that the result **NO ALC.** is displayed.

7 Storage

- Do not permanently store the device in its delivery packaging.

 Maintain the charge level of the rechargeable batteries.

Charge to at least 50 % every 6 months if not used for a prolonged period of time to avoid shortening the life span.

8 Disposal

This product must not be disposed of as household waste. This is indicated by the adjacent icon.

You can return this product to Dräger free of charge. For information please contact the national marketing organisations and Dräger.

Batteries must not be disposed of as household waste. They are therefore marked with the adjacent symbol. Dispose of batteries and rechargeable batteries at battery collection centers as specified by the applicable regulations.

9 Technical data

Measurement principle

Electrochemical sensor

Measuring range

Breath alcohol concentration	0.00 to 3.00 mg/L (mass of ethanol per minimum volume of breath at 34 °C and 1013 hPa)
Blood alcohol concentration	0.00 to 6.00 ‰ (mass of ethanol per blood volume or mass of blood at 20 °C and 1013 hPa)
A conversion factor of 2100 is used to calculate the blood alcohol concentration using the breath alcohol concentration data (factory setting, country-specific conversion factors can be configured).	
Measuring units	
mg/L, µg/100 mL, µg%, µg/L, g/210 L, g/L, ‰, ‰, mg/100 mL, mg%, mg/mL	
Ambient conditions	
During operation	-10 to +55 °C 10 to 93 % r. h. (non-condensing) 62 to 106 kPa
In storage	-20 to +60 °C 15 to 75 % r. h. (10 to 93 % r. h. <48 h)
Sampling	
Minimum volume of breath	>1.2 L (default)
Breath flow	>10 L/min (default)
Breath duration	>1.6 s (default)
Measurement accuracy	
Standard deviation of repeatability using standard alcohol	
Up to 0.50 mg/L	0.008 mg/L
≥0.50 mg/L	1.7 % of the measured value
Up to 1.00 ‰	0.017 ‰ or
≥1.00 ‰	1.7 % of the measured value
	Depending on which value is higher
Sensitivity drift	Typically 0.4 % of the measured value per month

Wait time until next measurement can be taken following a previous measurement of:

0.25 mg/L 20 s	0.50 ‰ 20 s
0.50 mg/L 40 s	1.00 ‰ 40 s
1.00 mg/L 80 s	2.00 ‰ 80 s
>1.50 mg/L 120 s	>3.00 ‰ 120 s

Adjustment	Recommended: every 12 months
Dimensions	58 mm × 170 mm × 32 mm (W × H × D)
Weight	approx. 245 g
Power supply	

Lithium-ion battery	Permanently installed in the device
---------------------	-------------------------------------

Interface	USB-C
------------------	-------

Classification in accordance with Regulation (EU) 2017/745

Measuring device	Class I with measuring function
Mouthpieces	Class I

LED risk group	2
-----------------------	---

Protection class	IP 54
-------------------------	-------

UMDNS code	17-475
-------------------	--------

DIMDI registration number	DE/CA01/102/XXX
----------------------------------	-----------------

CE marking	Electromagnetic compatibility as per IEC 60601-1 and IEC 60601-1-2
-------------------	--

The device meets the requirements of EN 15964. It can be configured in accordance with the standard. Display of measured values if configured in accordance with EN 15964:

Measured value	Display
0.0 mg/L to 0.03 mg/L	0.0 mg/L
>0.03 mg/L	Measured value

10 Order list

Designation and description	Order no.
Mouthpiece with exhalation non-return valve (100 pieces)	3705618
Wrist strap	3700416
Charging cradle	3700420
USB-C cable	3700418
Power supply unit	3701090
Dräger Mobile Printer BT	3700421
Standard ethanol solution 1.21 g/L (1 x 500 mL)	6728838

11 Annex

11.1 Glossary

Unit	Explanation
°C	Degrees Celsius; unit for temperature
°F	Degrees Fahrenheit; unit for temperature
hPa	Hectopascal; unit for ambient pressure
% r. h.	Percent relative humidity; relative atmospheric humidity
h	Unit for hours
‰	Per mill; unit for blood alcohol concentration
mg/L	Milligram per liter; unit for breath alcohol concentration

 Manufacturer
Dräger Safety AG & Co. KGaA
Revalstraße 1
D-23560 Lübeck
Germany
+49 451 8 82-0

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Regulation (EU) 2017/745 on medical devices

9300089 – 4754.021 me
© **Dräger Safety AG & Co. KGaA**
Edition: 05 – 2021-11
Subject to alterations

www.draeger.com