AlcoConnect CA8000

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TABLE OF CONTENTS

Intended use.....	13
Use	14
Preparing the measurement.....	15
Device description	15
Power supply	16
Operation.....	16
Errors and malfunction	19
Important notes.....	20
Specifications	21
Precautions.....	22
Accessories	22
Warranty	23

INTENDED USE

The AlcoConnect CA8005 is designed to measure the alcohol concentration in the human breath.

Commercial and frequent use will damage the device.

The measurement must not be used for any therapeutic measure.

Subject to technical changes.

USE

The device has been calibrated before delivery.

If the calibration date is more than 6 months ago, we recommend the recalibration of the device, e.g. by the ADCS service.

After the last alcohol intake, wait for about 15–20 minutes before taking a measurement. The residual alcohol in the mouth falsifies the measurement!

If aromatic drinks (e.g. fruit juice), alcoholic mouth sprays, medical liquids, and tobacco smoke are ingested, or after belching or vomiting, the results may be falsified.

A mouth rinse with water or non-alcoholic liquids does not replace the waiting time!

Do not use alcoholic cleaning agents or disinfectants for cleaning and disinfection.

Only use non-alcoholic disinfectants.

Make sure that no liquid seeps into the mouthpiece nozzle of the device.

The device must not be stored in the direct vicinity of alcoholic detergents and disinfectants (e.g. glass cleaner). The evaporating alcohol diffuses into the device and falsifies the measurement.

Only use mouthpieces with a non-return valve to prevent air from being sucked in from the device.


The mouthpiece is a disposable article which must be disposed after use. If used multiple times the device will be damaged by moisture and dust.

PREPARING THE MEASUREMENT

Do not drink alcohol at least 15 minutes before starting the measurement.


Use the device only at temperatures between 10 °C and 40 °C or 50 °F and 104 °F.

DEVICE DESCRIPTION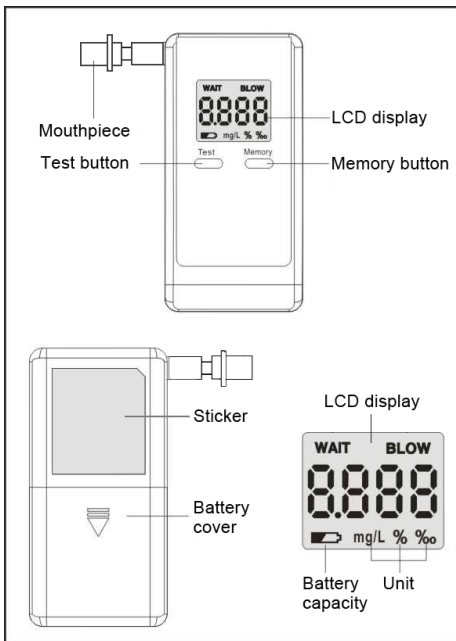

POWER SUPPLY


Slide back the battery cover on the back of the device and insert two AAA batteries (2 pieces).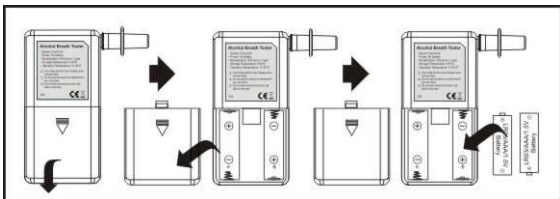


If the power is too low, the device will display a battery icon . The device may not measure correctly. Insert new batteries.

OPERATION


Test mode: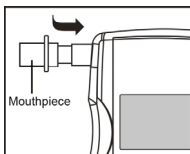

Insert a mouthpiece into the mouthpiece nozzle of the device.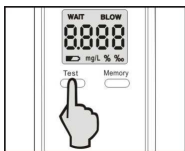

Press and hold the **Test** button until the device switches on. Release the **Test** button after the device has been switched on.

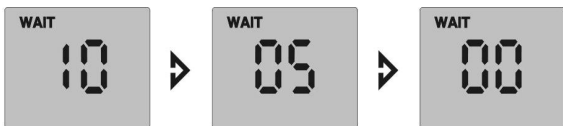The display shows **WAIT**; the device counts down from 10 to 0.

The display shows **BLOW** and a flashing **C**.

Take a deep breath and blow evenly into the mouthpiece as long as you hear the signal tone (about 6 seconds).

The device analyses the breath sample. **WAIT** appears on the display and a **0** flashes.


After the analysis, your test result is displayed on the screen for 20 seconds.

If not blown in correctly, **Err** is displayed. The device switches off after the error has been displayed. Please read the chapter „Errors and malfunctions“.

Memory mode: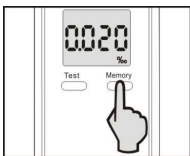

Press the **Memory** button to view the last 5 measurements. These are displayed for 20 seconds, after which the unit switches off.

ERRORS AND MALFUNCTIONS

ERROR	DESCRIPTION	DISPLAY
No display	<ul style="list-style-type: none"> – Press and hold the TEST button until the device has been switched on – Check the battery 	
Auto off	<ul style="list-style-type: none"> – Low battery voltage – Period of the test phase exceeded 	
Error	<ul style="list-style-type: none"> – Blowing-in unevenly or interrupted – Blowing-in time < 6 seconds – Blowing-in too weak – No blowing during BLOW mode – Sudden temperature change 	
	<ul style="list-style-type: none"> – Low battery indicator: Batteries must be replaced (use alkaline batteries) 	
Alarm tone (beep)	<ul style="list-style-type: none"> – Blow-in mode (continuous tone) – Error message (interrupted tone) – Alcohol measured value ≥ 0.5 ‰ (interrupted tone) – > 500 measurements (see counter when switching on) 	
HI	<ul style="list-style-type: none"> – Blood alcohol level above display range: Display range: 2,00 ‰ BAC 	

IMPORTANT NOTES

Do not drink alcohol at least 15 minutes before testing.

Blow evenly into the mouthpiece until the signal tone stops.

If the blowing-in is too short or interrupted, **ERR** (error) is displayed.

If the temperature changes considerably, wait a few minutes before using the device.

Only use the device in a temperature range of 10 °C to 40 °C (50 °F to 104 °F).

To ensure that the measurement results are within the specified accuracy range, the device should be calibrated every 6 months.

Never store the device in a room with corrosive gases.

Never store the device under sunlight.

Avoid damage to the device by shocks or dropping.

SPECIFICATIONS

European fuel cell sensor

Blow-in time: 6 seconds

Blow-in flow: 10l/min

Display range: 0.000-2.000 ‰ BAC

HI Display: if measured value ≥ 2 ‰

Accuracy: ± 0.05 ‰ BAC at 0.50 ‰ BAC

Memory: 5 measured values

Mouthpieces included: 5 pieces

Warm-up time: 10 seconds

Analysis time: 10 seconds

Recovery time before next test: < 60 seconds

Audible alarm/change of display colour: ≥ 0.50 ‰ BAC

4-digit display with status and backlight

Power supply: AAA batteries 1,5 V (alkaline batteries)

Low battery indicator

Automatic switch-off function after 1 minute

Display of measured values: %, ‰, mg/L

Operating temperature: 10 °C to 40 °C (50 °F to 104 °F)

Storage temperature: 0 °C to 50 °C (32 °F to 122 °F)

Size: 115 x 60 x 23 mm

Calibration interval: 6 months

PRECAUTIONS

Blood alcohol content (BAC) may change minutes after the measurement.

To avoid inaccurate measurements, the breathalyser should be calibrated every 6 months.

The device is not intended to determine whether you are fit to drive.

The test results of the breathalyser cannot be used in court.

The test results cannot be used against official measurements.

ACCESSORIES

Mouthpieces (5 pieces)

WARRANTY

The statutory warranty rights are applicable.

As a consumer, you are requested to promptly check the product for completeness, visible defects and transport damage as soon as it is delivered, and promptly disclose your complaints to us and the shipping company in writing. Even if you do not comply with this request, it shall have no effect on your legal warranty claims.

Insofar as you are a business, the following difference applies to the aforementioned warranty regulations:

- It is understood that the details provided by us and the product description provided by the manufacturer are the only things that represent the properties and condition of the product in question. Other advertisements, blurbs and statements issued by the manufacturer are not considered to be representative of the properties and condition of the said product.
- If the goods are found to be faulty, we shall reserve the right to repair the goods or deliver replacements. If the defect is not removed, you can demand a reduction in the price or withdraw from the contract at your discretion. The defect removal is applicable after a failed second attempt, unless the circumstances prove otherwise, in particular due to the nature of the object and/or defect or other conditions. In case of repair, we must not bear the additional costs, which arise from the transfer of the item to a place other than the place of fulfillment, as far as the transfer does not correspond to the intended use of the item.
- The warranty period amounts to a period of one year after delivery of the product. The reduction in time-limit does not apply:
 - to damages culpably attributable to us arising from injury to life, limb or health and for other damages caused by wilful intent or gross negligence;
 - insofar as we have wilfully concealed the defect or accepted a warranty for the quality of the goods;
 - to goods which are used for a building in accordance with their normal use instructions and whose defects were caused by this;
 - for statutory recourse claims which you have against us in connection with warranty rights.

Vertrieben durch/distributed by:

ADCS

Alcohol & Drug Checking Systems

Borsigstraße 14

23560 Lübeck

Germany

Tel./Phone: +49 (0)451 61 91 77 51

Fax: +49 (0)451 61 91 77 52

E-Mail: info@adcs-germany.com

www.adcs-germany.com

CA8000 Version 11-2020