

AirBP 2

Blood Pressure Monitor



User Manual (EN)

Benutzerhandbuch (DE)

Manuale d'uso (IT)

Manual de usuario (ES)

Manuel de l'utilisateur (FR)

Contents

User Manual.....	English 1-16
Benutzerhandbuch.....	Deutsch 17-35
Manuale d'uso.....	Italiano 36-54
Manual de usuario.....	Español 54-71
Manuel de l'utilisateur.....	Français 72-89

User Manual

Contents

1. The Basics.....	2
2. Introduction.....	4
2.1 Device Description.....	4
2.2 Intended Use	5
2.3 Contraindications	5
2.4 Symbols.....	5
3. Using the Monitor	6
3.1 Charging the Battery.....	6
3.2 Beep indications	7
3.3 Applying the Arm Cuff.....	7
3.4 How to Sit Correctly	8
3.5 Connect to the App.....	8
3.6 Measurement.....	9
3.7 After the Test.....	9
3.8 Reviewing the History	10
3.9 Measurement without App	11
3.10 Trouble Shooting	11
4. Accessories	12
5. Specifications	12
6. Maintenance	14
6.1 Maintenance	14
6.2 Cleaning	14
6.3 Storage.....	15
7. Electromagnetic Compatibility	15

1. The Basics

This manual contains the instructions necessary to operate the product safely and in accordance with its function and intended use. Observance of this manual is a prerequisite for proper product performance, correct operation, and ensures patient and operator safety.

Safety

- Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
- Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.
- Do not use in a location with moisture, or a location where water may splash on the device. This may damage the device.
- Do not disassemble or attempt to repair the monitor or components. This may cause an inaccurate reading.
- To measure blood pressure, the arm must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a

temporary red mark to the arm. This condition will appear especially when measurement is repeated successively.

- Stop using this monitor and consult your physician if you experience skin irritation or discomfort.
- Consult your physician before using this monitor on an arm with an arterio-venous (A-V) shunt.
- Consult your physician before using this monitor if you have had a mastectomy or lymph node clearance.
- Consult your physician before using the monitor if you have severe blood flow problems or blood disorders as cuff inflation can cause bruising.
- People who have a severe circulatory deficit in the arm must consult a physician before using the device, to avoid medical problems.
- Do not self-diagnose the measurement results and start treatment by yourself. Always consult your physician for evaluation of the results and treatment.
- Do not apply the cuff on the injured arm or the arm under medical treatment.
- Do not use the device with other medical electrical (ME) equipment simultaneously.
- Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
- Do not use the device in the area of HF surgical equipment, MRI, or CT scanner, or in an oxygen rich environment.

- Use only the approved arm cuff for this device. Use of other arm cuffs may result in incorrect measurement results.
- Rest for at least 5 minutes before taking the measurement.
- Remove tight-fitting or thick clothing from your arm while taking a measurement.
- Remain still and do not talk while taking a measurement.
- If the patients' arm is outside the specified circumference range (22-42cm) that may result in incorrect measurement results.
- The device is not intended for use with neonatal, pregnant, including pre-eclamptic, patients.
- When choosing a third-party charging adaptor, select one that complies with IEC 60950 or IEC 60601-1.
- Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

2. Introduction

2.1 Device Description

The blood pressure monitor includes two models, BP1 and BP1A. It uses the oscillometric method to measure blood pressure with Bluetooth technology. The monitor is comprised of a cuff, a main unit, a rubber hand-pump, an exhaust valve and an air hose. The blood pressure data is displayed, stored and reviewed including the systolic and diastolic

blood pressure, as well as pulse rate in an application which is installed on a smart phone.







2.2 Intended Use







The subject device is intended to measure diastolic and systolic blood pressure, as well as pulse rate of the adult population in home and hospital facilities by using a non-invasive, oscillometric technique with a single upper arm cuff (22-42 cm).

2.3 Contraindications

- The use of this device is prohibited in ambulatory environments.
- The use of this device is prohibited on aircraft.

2.4 Symbols

Symbol	Description
	Manufacturer
	Date of manufacture
SN	Serial number
	Indicates a medical device that is not to be disposed of as unsorted municipal waste.
	Follow Instructions for Use.
	Type BF Applied Part
	No alarm system

	MRI unsafe. Presents hazards in all MR environments as device contains strongly ferromagnetic materials.
IP22	Resistant to liquid ingress
	CE marking
	Authorized representative in the European community
	UKCA marking
	Authorized Representative in the United Kingdom
	This product complies with the rules and regulations of the Federal Communication Commission.

3. Using the Monitor

Download the App **AirBP**

iOS: App Store

Android: Google Play

Note: if you've downloaded the App before, please update to the latest version.

3.1 Charging the Battery

Charge the monitor by using the USB cable to connect a USB charger or to the PC USB Port. It

takes 2 hours to fully charge the monitor




In the App, you can check the battery level.

Note: The device cannot be used while charging.

3.2 Beep indications

two short beeps	Stop pumping
One long beep	End of measurement
Three short beeps	Pump pressure is too high (above 300mmHg)

3.3 Applying the Arm Cuff

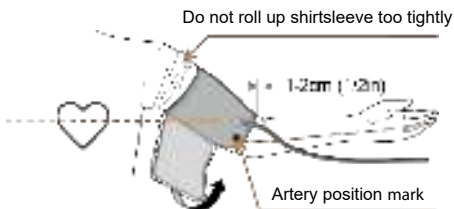

Wrap the cuff around the upper arm, about 1 to 2cm above the inside of the elbow, as shown.

- Place the cuff directly against the skin, as clothing may cause a faint pulse and result in a measurement error.
- Constriction of the upper arm, caused by

rolling up a shirtsleeve, may prevent accurate readings.

- Confirm that the artery position mark is lined up with the artery.

3.4 How to Sit Correctly

To take a measurement, you need to be relaxed and comfortably seated. Sit in a chair with your legs uncrossed and your feet flat on the floor. Sit with both your back and arm being supported. Place your arm on a table so the cuff is level with your heart.



3.5 Connect to the App

1. Turn on the **Phone's Bluetooth**.
2. Press the button to **power on the monitor**.
3. **Open the Air BP APP** on your smartphone. The App will search for the device. Choose your device "AirBP xxxx" in the App.

Note:

- *Keep the monitor and the phone within 1.5 meters of each other.*

- *DO NOT pair in your phone settings.*
- *The APP can be downloaded in **Google Play** or **APP Store** with the name "AirBP", which requires a smartphone.*

3.6 Measurement

Make sure your phone speakers are turned on and the volume is loud enough. Follow the text and voice guide in the App to use.

In App->Measure,

- Follow the instructions to inflate the cuff by squeezing the pump at the rate indicated in the App.
- **Stop pumping** according to instructions of the App, hold the pump without squeezing it, and wait for the further instructions. In some cases, if the App senses that the pressure in the cuff is not high enough for use, then the user will be instructed to **"pump again"**.

Note: *During use you should keep your body still. Otherwise, the blood pressure readings may be inaccurate.*

Stop pumping when the screen of the monitor show "Out of Rang" and beep 3 short beeps (pressure is too high).

3.7 After the Test

- The blood pressure readings will appear on the App interface when the test has finished. You need to manually press the exhaust valve



to **deflate the cuff**. If you forget to deflate the cuff manually after use, the monitor will automatically deflate the cuff, which takes less than 30s for full automatic deflation.

- On the results page, you can add user names and ID's to help you manage multiple users data, add notes, share results or delete them.
- If the monitor detects an irregular heartbeat, then the corresponding symbol will display below the PR value.

Note: The device will turn off automatically after two minutes if not working.

3.8 Reviewing the History



History		
Date and Time	User	PR
10/20/2019 10:00	1 user	68
10/20/2019 10:01	1 user	66
10/20/2019 10:02	1 user	65
10/20/2019 10:03	1 user	60
10/20/2019 10:04	1 user	62
10/20/2019 10:05	1 user	62
10/20/2019 10:06	1 user	67
10/20/2019 10:07	1 user	65

3.9 Measurement without App

After you have learned how to use the App for measurement. You can also perform measurement without connecting to the app when you do not need to store data.

1. Refer to section 3.3 and 3.4 to wear the cuff.
2. Inflate the cuff by squeezing the pump at the similar rate you have learned in the App.
3. **Stop pumping** when the monitor beep twice, hold the pump without squeezing it, and wait for the further instructions. In some cases, if the monitor senses that the pressure in the cuff is not high enough for use, then the user will be instructed to "**Pump**" again.
4. **After the test.** The blood pressure readings will appear on the screen of the monitor when the test has finished. You need to manually press the exhaust valve to **deflate the cuff**.

Note:

- *If you want to start a new measurement, you can short press the power button of the monitor. Or pump*
- *When the monitor is not connected to App for measurement, the results will not be saved.*

3.10 Trouble Shooting

Problem	Possible Cause	Recommended Action
---------	----------------	--------------------

The monitor cannot be connected to the phone	The phone Bluetooth is OFF	Turn on the phone Bluetooth in the settings menu.
	The phone doesn't support the Bluetooth 4.0 BLE	Change to a compatible phone.
The monitor doesn't respond when pressing the button.	The monitor is running in an unexpected status.	Reset the device by pressing and holding the button for 5s.
Cannot get blood pressure readings.	The measurement is interrupted by arm movement or unexpected bulb squeezing during use	Keep the arm still and don't squeeze the bulb during deflation.
	There is pressure leakage.	Check the hose connection to see if it is loose.

4. Accessories

Model	Description
CU-10	Adult, arm size 22-42cm
540-00240-00	MICRO USB charger cable

Arm size: The circumference at the biceps.

5. Specifications

Classifications

Protection against electrical shock	Type BF	
Environmental		
Item	Operating	Storage
Temperature	5 to 40°C	-25 to 70°C
Relative humidity (non-condensing)	10% to 95%	10% to 95%
Barometric	700 to 1060 hPa	700 to 1060 hPa
Degree of dust & water resistance	IP22	
Physical		
Size	68mm(long)×25mm(diameter) (main unit)	
Weight	Less than 30 g (main unit)	
Cuff size	Adult cuff: 22-42cm	
Wireless connectivity	Built-in Bluetooth 4.0 BLE	
Power Supply		
Charger input	Micro USB, DC5V	
Battery type	Rechargeable lithium-polymer battery	
Estimated charging time	Approximately 2 hours	
Blood Pressure		
Technology	Oscillometric method	
Cuff pressure Range	0 – 300 mmHg	
Pressure measurement accuracy	±3mmHg	
Pulse rate range	40 to 200 bpm	
Pulse rate accuracy	±2 bpm	
Mobile APP		

APP function	Guided measurement, display results, store and share results
IOS software / hardware	IOS 12.0, iPhone
Android software / Hardware	Android 6.0 or above, mobile phone with Bluetooth 4.0BLE
Bluetooth RF	
Frequency range	2.402 – 2.480 GHz
Durable period for Monitor	
Expected service life	2 years

6. Maintenance

6.1 Maintenance

To protect your monitor from damage, store the monitor and the components in a clean, safe location.

Caution: DO NOT disassemble or attempt to repair this

monitor or other components. This may cause inaccurate blood pressure readings.

6.2 Cleaning

- Do not use any abrasive or volatile cleaners.
- Use a soft dry cloth or a soft cloth moistened with mild (neutral) detergent to clean your monitor and the arm cuff and then wipe them with a dry cloth.
- When electrodes are dirty, use a soft cloth or cotton swab moistened with an alcohol-based sanitizer to clean the electrodes.
- Do not use gasoline, thinners, or similar

solvents to clean your monitor and arm cuff or other components.

6.3 Storage

Keep your monitor and other components in the storage case when not in use.

- Store your monitor and other components in a clean, safe location.
- Do not store your monitor and other components in locations exposed to extreme temperatures, humidity, direct sunlight, dust, or corrosive vapors, such as bleach.

Disposal



7. Electromagnetic Compatibility

The device meets the requirements of EN 60601-1-2.



- Using accessories other than those specified in this manual may result in increased electromagnetic emissions or decreased electromagnetic immunity for the device.
- The device or its components should not be used adjacent to or stacked with other equipment.

- The device needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided below.
- Other devices may interfere with this device even though they meet CISPR requirements.
- When the inputted signal is below the minimum amplitude provided in the technical specifications, erroneous measurements could result.
- Portable and mobile communication equipment may affect device performance.
- Other devices that have RF transmitters or sources may affect this device (e.g. cell phones, PDAs, and PCs with wireless functionality).

The EMC table information is listed on our website:
http://api.viatomtech.com.cn/documents/2017/emc_en.pdf