snorflex®

CPAP CLEANER PRO

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Hersteller

Huizhou Gold Rose Technology Co.,Ltd
5 / F, Huaheng Industrial Park, 12 Songyang Road, Songshan Industrial Zone,
Zhongkai, Huizhou City, Guangdong Province, P.R.China

SNORFLEX®

CPAP CLEANER PRO

User Manual English

Congratulations on your new CPAP CLEANER PRO!

The CPAP CLEANER PRO supports you in the daily disinfection of your CPAP accessories so that bacteria, mold and germs have no breeding ground. The CPAP CLEANER PRO offers you three different modes: disinfection with UV-C, disinfection with ozone and disinfection with UV-C and ozone. In this small manual you will find step-by-step instructions and everything you need to know about your new product. **Please read and comply with all of the instructions and warnings provided in this manual before using your CPAP CLEANER PRO.**

We hope you enjoy your new CPAP CLEANER PRO and we will be happy to answer any questions you may have.

EASY. FAST. EFFICIENT. ENVIRONMENTALLY FRIENDLY.

Package content

- 1 × CPAP CLEANER PRO, model Q8
- 1 × adapter
- 1 × tube adapter
- 1 × user manual
- 6 × activated carbon filter made of cotton

CPAP CLEANER PRO

adapter

tube adapter

user manual

activated carbon filter
made of cotton



Specifications

size	230 × 230 × 185 mm	store temperature	-20 ~ +55°C
weight	1,56 kg	operation temperature	-5 ~ +45°C
input	100-240 V ~ 50/60 Hz	power	10W
output	5V2A		



Information on ozone and UV-C intensity

Ozone	UV-C
~16 ppm/min When operating in a 12 square meter room (ceiling heighth: 2.5 meters), the average ambient ozone concentration in the room during the cleaning operating cycle is less than 0.05 parts per million (ppm). The maximum workplace concentrations of ozone are 0.1 ppm for the EU, Great Britain, USA and Canada.	- Wavelength: 254 nm - Intensity: ~75 mj/cm² - Lifetime of the UV-C beads: ~10.000 hours

Functions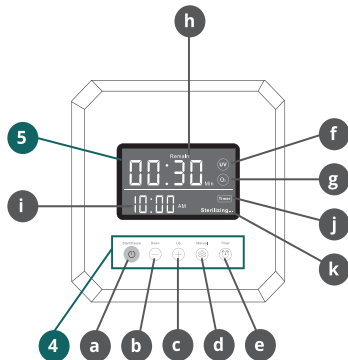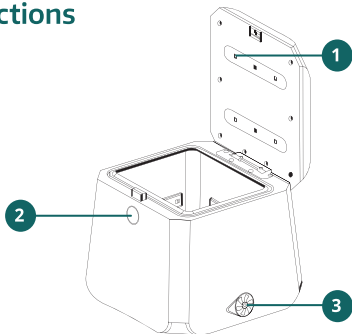

1. UV-C light
2. Open button
3. External sealing lid
4. Key panel:
 - a) "Start/Pause" button
 - b) button "-"
 - c) button "+" for time setting
 - d) "Manual" button for mode selection
 - e) "Timer" button (time programming)
5. Display panel:
 - f) UV indicator
 - g) Ozon indicator
 - h) remaining disinfection time
 - i) clock
 - j) manual/timer mode
 - k) working indicator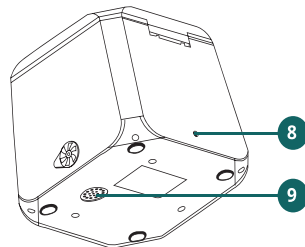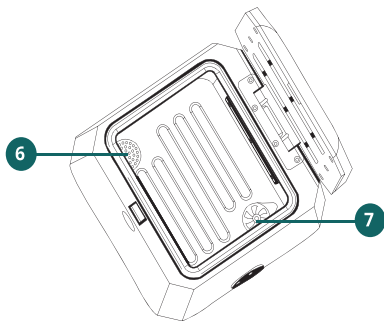

- 6. Inner activated carbon filter
- 7. Internal sealing lid
- 8. DC input
- 9. Outer activated carbon filter

Switch on the CPAP CLEANER PRO / interrupt the current program


To switch on the device or to interrupt the running program, briefly press the “Start/Pause” button.

How to turn off the CPAP CLEANER PRO

Press the “Start/Pause” button for 3 seconds to turn off the machine. When not in use, the device switches off automatically after about 10 minutes.


How to set the clock

1. Press and hold the timer button and press the buttons “+” and “-” to adjust the hours.
2. Then press the timer button to switch to minutes and press the buttons “+” and “-” to adjust the minutes.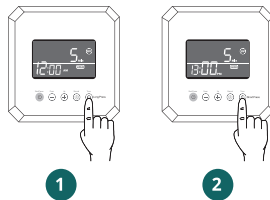

How to set the disinfection timer mode: automatic disinfection at a specific time

1. Set the clock to ensure that it is consistent with your local time (the clock must be set again after each power cut/failure).
2. Select one of the five disinfection modes mentioned on page 10 and follow the instructions given there to set the disinfection modes and the duration of the disinfection.
3. Press the button "Timer" until the symbol "Timer" lights up on the display. This indicates that the timer mode is set.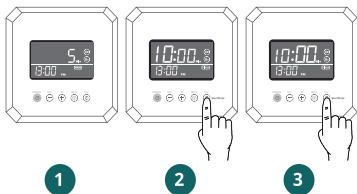

After you have used your CPAP device, you can place or connect the items to be disinfected (mask, hose, etc.) into your CPAP CLEANER PRO. The disinfection starts automatically every day at the time you have programmed.

Overview table disinfection modes

Function	Application	Internal sealing lid (Fig. page 7, no. 7)	External sealing lid (Fig. page 6, no. 3)	Disinfection times	recommended disinfection times
UV-C disinfection (in the chamber)	Disinfection of CPAP mask, other objects	closed	closed	5, 10 and 15 minutes	10 and 15 minutes
Ozone disinfection (in the chamber)	Disinfection of CPAP mask, hose, other objects	open	closed	20, 30 and 40 minutes	20 minutes for CPAP mask
UV-C & ozone disinfection (in the chamber)	Disinfection of CPAP mask, hose, other objects	open	closed	25 until 55 minutes**	30 minutes (10 min. UV-C + 20 min. ozone disinfection)
Ozone disinfection (outside/external)	Disinfection of CPAP device* and hose	closed	open	20, 30 and 40 minutes	30 minutes for CPAP device/hose*
UV-C & ozone disinfection (in the chamber) & ozone disinfection (outside/external)	Disinfection of a) CPAP mask & other objects (in the chamber) b) CPAP device* and hose (external)	open	open	25 until 55 minutes**	40 minutes (10 min. UV-C + 30 min. ozone disinfection)

* The CPAP device itself should only be cleaned with ozone 1-2 times per month.

** The disinfection times of UV-C and ozone add up. The CPAP CLEANER PRO displays first the UV-C disinfection time. After this, the CPAP CLEANER PRO switches to the ozone disinfection time.

Ozone disinfection process


Ozone disinfection times	Ozone disinfection process
20 minutes	- 5 minutes supply of ozone - 5 minutes cycle of 20 seconds of ozone supply and 40 seconds of air supply - 10 minutes of air supply
30 minutes	- 5 minutes supply of ozone - 15 minutes cycle of 20 seconds of ozone supply and 40 seconds of air supply - 10 minutes of air supply
40 minutes	- 10 minutes supply of ozone - 15 minutes cycle of 20 seconds of ozone supply and 40 seconds of air supply - 15 minutes of air supply

Note: Provide adequate ventilation during ozone disinfection so that ozone residues and odors can be completely removed. Please also note the safety instructions on pages 17 to 19.

Disinfection modes

1. UV-C disinfection (in the chamber) – **INTERNAL** and **EXTERNAL** sealing lids are closed


- Open the lid of the CPAP CLEANER PRO and place the objects (CPAP mask, etc.) into the chamber. Then close the lid.
- Press the button "Start/Pause" to turn on the device.
- Press the button "Manual" repeatedly until the symbol "UV" appears on the display.
- Press the buttons "+" and "-" to manually choose the duration of the UV-C disinfection.
The standard UV-C disinfection times are 5, 10 and 15 minutes.
- Press the button "Start/Pause" to start the disinfection.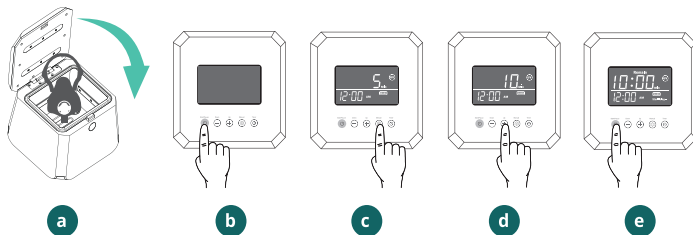

2. Ozone disinfection (in the chamber) –

INTERNAL sealing lid is open, **EXTERNAL** sealing lid is closed

- Open the lid of the CPAP CLEANER PRO and the internal sealing lid. Then put the objects into the chamber (CPAP mask, hose, etc.) and close the lid.
- Press the button "Start/Pause" to turn on the device.
- Press the button "Manual" repeatedly until the symbol " O_3 " appears on the display.
- Press the buttons "+" and "-" to manually choose the duration of the ozone disinfection.
The standard ozone disinfection times are 20, 30 and 40 minutes.
- Press the button "Start/Pause" to start the disinfection.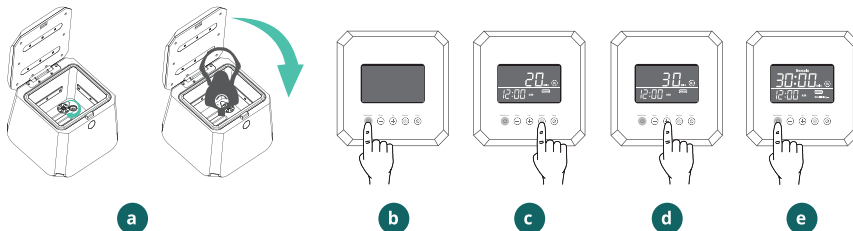

3. UV-C & ozone disinfection (in the chamber) –

INTERNAL sealing lid is open, **EXTERNAL** sealing lid is closed


- a) Open the lid of the CPAP CLEANER PRO and the internal sealing lid. Then put the objects into the chamber (CPAP mask, hose, etc.) and close the lid.
- b) Press the button "Start/Pause" to turn on the device.
- c) Press the button "Manual" repeatedly until the symbol "UV" appears on the display.
- d) Press the buttons "+" and "-" to manually choose the duration of the UV-C disinfection. The standard UV-C disinfection times are 5, 10 and 15 minutes.
- e) Then press the button "Manual" again. The symbol "O₃" appears on the display.
- f) Press the buttons "+" and "-" to manually choose the duration of the ozone disinfection. The standard ozone disinfection times are 20, 30 and 40 minutes.
- g) Then press the button "Manual" again until the characters "UV" and "O₃" appear on the display at the same time. The device displays the UV-C disinfection time. After this, the device switches to the ozone disinfection time.
- h) Press the button "Start/Pause" to start the disinfection.

Please note that the times of UV-C and ozone disinfection are added, e.g. you select 10 minutes for UV-C disinfection and 30 minutes for ozone disinfection, then the total disinfection time is 40 minutes.

4. Ozone disinfection (outside/external) –

INTERNAL sealing lid is closed, **EXTERNAL** sealing lid is open

- Open the external sealing lid and connect the hose to the CPAP CLEANER PRO by using the tube adapter supplied. Please make sure that the internal sealing lid is closed. Then close the main lid.
- Press the button “Start/Pause” to turn on the device.
- Press the button “Manual” repeatedly until the symbol “O₃” appears on the display.
- Press the buttons “+” and “-” to manually choose the duration of the ozone disinfection.
The recommended ozone disinfection times are 20, 30 and 40 minutes.
- Press the button “Start/Pause” to start the disinfection.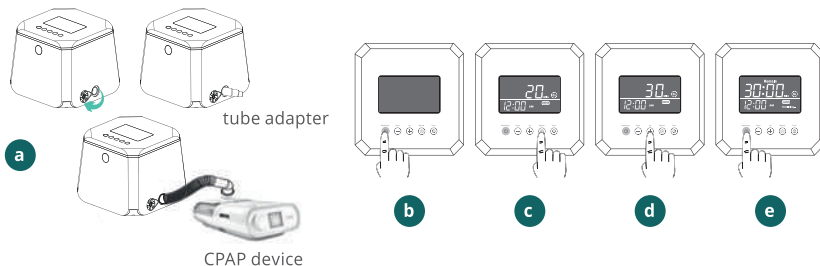

5. UV-C and ozone disinfection (internal chamber) as well as ozone disinfection (outside/external) – **INTERNAL** and **EXTERNAL** sealing lids are open

- Open the lid of the CPAP CLEANER PRO and the internal sealing lid. Then put the objects into the chamber (CPAP mask, etc.).
- Open the external sealing lid and connect the tube to the CPAP CLEANER PRO by using the tube adapter supplied. Then close the lid.
- Press the button "Start/Pause" to turn on the device.
- Press the button "Manual" repeatedly until the symbol "UV" appears on the display.
- Press the buttons "+" and "-" to manually choose the duration of the UV-C disinfection.

The recommended UV-C disinfection times are 5, 10 and 15 minutes.

- Then press the button "Manual" again. The symbol "O₃" appears on the display.
- Press the buttons "+" and "-" to manually choose the duration of the ozone disinfection.


The standard ozone disinfection times are 20, 30 and 40 minutes.

- Then press the button "Manual" again until the characters "UV" and "O₃" appear on the display at the same time.

The device displays the UV-C disinfection time. After this, the device switches to the ozone disinfection time.

- Press the button "Start/Pause" to start the disinfection.

Please note that the times of UV-C and ozone disinfection are added, e.g. you select 10 minutes for UV-C disinfection and 30 minutes for ozone disinfection, then the total disinfection time is 40 minutes.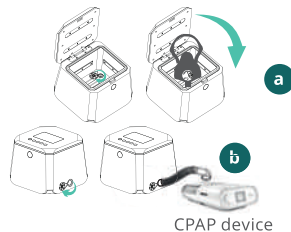

Safety Information

Important: Failure to comply with the instructions and warnings provided herein may result in inaccurate results and/or damage to the product itself or your health and safety (e.g. burns, shortness of breath, electric shock, injuries). No liability is assumed in the event of improper use/operation.

Warning notes:

- Do not place any living organisms inside the disinfecting chamber at any time.
The disinfecting chamber is only for non-living things.
- Keep your CPAP CLEANER PRO out of the reach of children.
- Do not use your CPAP CLEANER PRO in explosive air environments, near gas vapours, near other flammables or similar.
- Do not place any flammable or ignitable items anywhere near your CPAP CLEANER PRO.
- Do not operate or place your CPAP CLEANER PRO on or near hot surfaces such as ovens, heaters, or active fireplaces.
- Only store your CPAP CLEANER PRO indoors away from direct sunlight or chemicals.
- Do not move your CPAP CLEANER PRO or open the lid while the machine is running.
- Do not disassemble or repair the product yourself. If it is damaged, please contact your distributor during the warranty period and report the damage.
- Do not submerge your CPAP CLEANER PRO device in liquid or use chemical cleaners.
Keep your CPAP CLEANER PRO away from water sources including rain, bathtubs, sinks and pools. Wipe your CPAP CLEANER PRO with a damp cloth to clean the device. Do not soak or damage the filters during cleaning.


- Ozone has a strong odour and can cause headaches, nausea, coughing, respiratory and skin irritation, reduced lung function and inflammatory reactions in the respiratory tract. Therefore, always open a window during the ozone-mode operation and leave the room!
- Do not wear your CPAP mask immediately after ozone disinfection. Residual ozone may hurt your respiratory system or cause damage to health. The CPAP masks or other devices should be well ventilated for several hours (at least 2-3 hours) after ozone disinfecting cycle to prevent damage to health. Please wait until the ozone odour has completely disappeared. Our recommendation: Carry out the ozone disinfection in the morning after getting up, then you can safely use your CPAP mask and accessories at night.
- Do not breathe through your CPAP mask or hose while the ozone disinfection is in progress.
- Only run the ozone disinfecting cycle when windows are widely open and sufficient air exchange is guaranteed. Please leave the room during ozone disinfecting cycle.
- Do not inhale from the injection tube, lid surroundings or the injection tube outlet of the device.
- Keep the lid of the CPAP CLEANER PRO closed until the end of the sterilization/disinfection. Do not breathe in any remaining ozone gas that escapes when you open the lid.
- In case you smell strong ozone odour, please turn off the device and check it for any visible damage, such as cracks or an open sealing lid.
- People with acute or chronic respiratory conditions such as asthma, bronchitis and sensitive or irritated breathing passages (mouth, nose, throat) should consult their medical doctor before using the CPAP CLEANER PRO.
- The CPAP device should only be cleaned with ozone 1-2 times a month. CPAP masks and other items can be cleaned more often with ozone or UV-C.

- Do not look into the UV-C light and do not expose your skin to direct UV-C light radiation.
- Only UV-C light treatment is recommended for disinfection of small devices such as cell phones, jewelry, watches or similar items. Ozone treatment could impair the functionality of the devices, especially in the long run.
- Use the UV-C disinfection of the CPAP CLEANER PRO only for UV-C resistant objects.
- UV-C light can damage plastics, color pigments and paints. Organic paints may bleach out and plastics may become cloudy.
- Please only use the original adapter and accessories.
- In the event of power loss to your home or the CPAP CLEANER PRO device, the correct clock time and cycle start time has to be set.
- Please change the filter every two months to ensure the effectiveness of the ozone sterilization.
- When using ozone to disinfect your CPAP device, masks and tubes, please follow the instructions of the manufacturer of the CPAP device, masks and tubes for your safety.

Replace activated carbon filter

1. Remove the screw with a screwdriver.
2. Remove the filter cover.
3. Replace the old filter with the new filter.
4. Reassemble the filter cover and the screw.

Tip: It is recommended to replace the filter every 2 months.

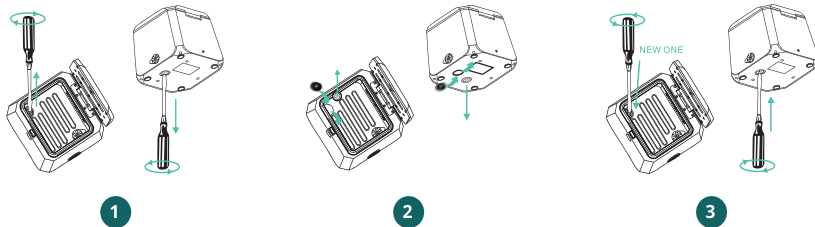Routine care

1. When not using your CPAP CLEANER PRO for a long time, please put it back in the packaging box.
2. Do not place the CPAP CLEANER PRO in damp rooms or in the sun.
3. When using the CPAP CLEANER PRO, please put it in a dry, clean and ventiated environment.
4. Do not drop or damage the CPAP CLEANER PRO to avoid damage to the eletronic system.
If cracks are noticed, stop using the CPAP CLEANER PRO and contact customer service.
5. Wipe your CPAP CLEANER PRO with a damp cloth. Do not use detergents. They may damage the surface of the product.

Manufacturer

Huizhou Gold Rose Technology Co.,Ltd

5 / F, Huaheng Industrial Park, 12 Songyang Road, Songshan Industrial Zone,
Zhongkai, Huizhou City, Guangdong Province, P.R.China

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Huizhou Qichengda Technology Co. Ltd.