Heart Health Monitor (ER1)

User Manual (EN)

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Manuale d'uso (IT)

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Contents

User Manual.....	English 1-17
Benutzerhandbuch.....	Deutsch 18-35
Manuale d'uso.....	Italiano 36-53
Manual de usuario.....	Español 54-71
Manuel de l'utilisateur.....	Français 72-89

Table of Contents

NOTICE.....	2
Introduction.....	2
Monitor Views.....	3
Getting Started.....	4
Wear ER1 Monitor.....	4
Start and Stop Recording.....	7
Install ECG Browser.....	7
Registration and Login.....	8
Make an Analysis.....	9
Warnings.....	10
Cleaning.....	13
Recycling.....	13
Troubleshooting.....	13
Appendix.....	14

NOTICE

It's not intended for medical use or treat. It's marketed as a consumer wellness product.

Introduction

Thank you for choosing ER1 heart health monitor (hereinafter ER1 monitor).


ER1 monitor is to acquire, record and store adult ambulatory ECG data. The ECG data from ER1 monitor can be automatically analyzed when uploaded to the accompanying PC software.

Composition

The ER1 monitor box contains:

- ER1 main unit
- A power/data cable for charging and exporting
- A USB drive with the analysis software (ECG browser)
- A pack of disposable electrodes (10 pieces)
- Chest strap

Monitor Views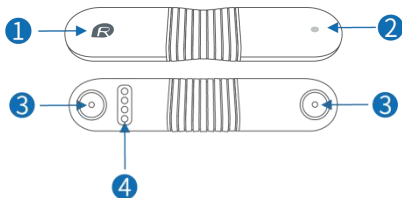

1 “R” symbol

Indicates this side should be placed rightwards.

2 LED indicator

With light	Light flashing	• Yellow light: Low battery • Flash every 5 seconds: Standby • Flash when applied: Synchronous with the heartbeat.
	On	• Green light: Fully charged • Yellow light: Charging
No light	/	Battery depleted.

3 Electrode button

It is to use with chest strap, disposable electrode or power/data cable.

4 Power contact

It is used to connect the power/data cable.

Getting Started


First Use

After removing the packing, **charge the device at first for activation.**

Charge Battery

This device uses a rechargeable lithium battery. Charge it through the following steps:

1. Connect the main unit with the power/data cable, as shown below.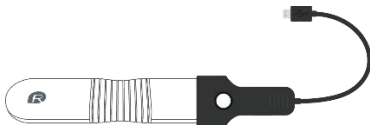

2. Connect the other end of the cable to the power source of 5V output/USB connector, like laptop or power adapter.

Wear ER1 Monitor


Before wearing the ER1 monitor, you need decide to use which way to apply it to your body. The device is designed for application through the following two ways:

- With electrodes
- With chest strap


You can choose one way to apply it to acquire ECG data.

With Electrodes

1. Take two accompanying disposable electrodes and remove their packing.
2. Connect the electrodes with the ER1 main unit, as shown below.

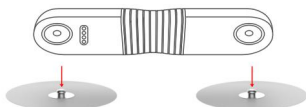
3. Apply the device to the left chest, as shown below. The “R” symbol is rightward and near the middle of sternal notch, and the other end is leftward and 45° lower than the horizontal level.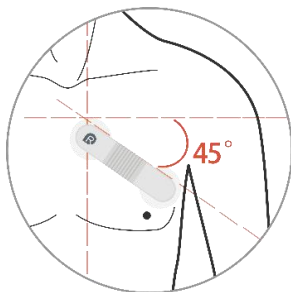

With Chest Strap


- 1 Take the accompanying chest strap and remove its packing.
2. Install the ER1 main unit to the strap, as shown below.

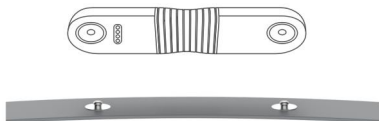3. Moisten the electrode on the strap contacted the body with clean water.

4. Adjust the chest strap length to suit your body.
5. Put on the strap with the “R” symbol at the right side, and then position the strap under the breast, as shown below.

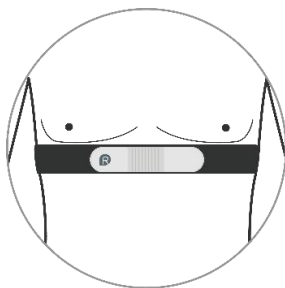Start and Stop Recording

Start Recording

Once you wear the ER1 monitor, the device starts acquiring ECG data.

Stop Recording

When 60 seconds elapse after you remove the ER1 monitor, the monitor stops acquiring ECG data and saves it.

Attention

The device can store 10 recordings, 30 hours in total. When the recordings exceed 10 pieces, or total time reaches 30 hours, the earliest data will be covered with a new one.

One single recording can only store 24 hours of data at most.

For the safety of your data, export it to the ECG Browser each time after you have measured ECG.

Install ECG Browser

When you want to analyze the acquired ECG data in your ER1 monitor, you need install the ECG browser in your PC or laptop. Then you can import the data into the ECG browser and make analysis online.

Follow the steps below to install the ECG browser:

1. Plug the USB drive accompanied to the USB connector on your PC or laptop.
2. Copy the file of *ECG Browser Setup vxx.exe* to your local drive.
3. Run the file.
4. Select the appropriate language.
5. Select the defaults until finish installation.

Registration and Login

When you first use this browser, you need register.


1. Run the  on the desktop on your PC or laptop.
2. In the lower part of the pop-up screen, select “Register now”.
3. Enter email address, verification code, and password.
4. Select “Agree to and accept Terms of Use and Privacy Policy”.
5. Select “Register now” button.

After you have registered, you just run  to log in each time you use this browser.

Make an Analysis

1. Make sure that your PC or laptop have connected to the Internet.
2. Connect the ER1 monitor to your PC or laptop with the power/data cable.
3. Log in the ECG Browser.
4. Select the “Download Data” button.
5. Create or select a user. Then the data will automatically be imported to the browser and shown in the task list.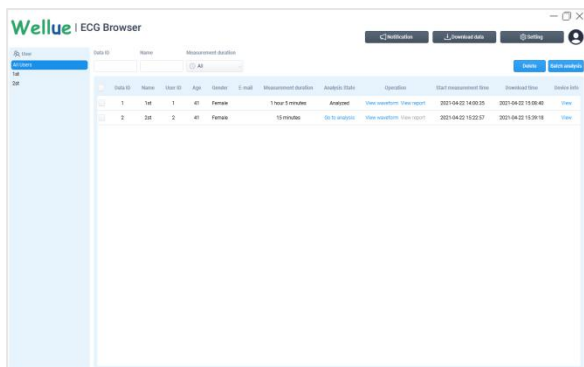

6. Select “Go to analysis”.
7. Wait till the analysis result returned.
8. Select “View Report”.

Additional Operations:

- Save the waveform data in PDF by selecting “Save as PDF” in the “View Waveform” window.
- Delete the recordings by selecting “Delete” button.
- Analyze the data in batch by selecting “Batch Analysis” button.
- Set the storage directory in “Setting” window.
- Change the password by selecting  icon.

Attention

The report and waveform data are saved in the default directory. If you change the storage directory, the previously stored recordings will not be seen in the task list.

The default directory is C:/User/AppData/Local/ECG Browser/Data.

Warnings

General

Before using this device, read this manual carefully and fully understand the warnings and risks.

Do not use acetone or other volatile solutions to clean the device.

Do not strongly collide or crush the device. If the casing is broken, stop using it.

Do not disassemble the device at will, otherwise it may cause machine malfunction or affect the normal operation of the device.

Keep the device out of the reach of children and pets.

The device should not be maintained while in use.

Analysis

The device is not intended to replace the medical diagnosis of a professional physician.

The results of this device are for reference only and cannot be directly used as a basis for clinical treatment.

The device cannot be used to diagnose a disease directly. Please consult your doctor.

The measurement results of this device cannot distinguish all diseases. If your body feels unwell, you should consult your doctor immediately, in addition to the measurement results of this device.

Do not self-diagnose and take medication based on the measurements of this device without consulting your doctor. In particular, do not take new medications without prior permission.

The device is not a substitute for professional heart or other organ function measurement equipment. Medical ECG measurement requires more professional and complete measurements.

We recommend that you record your ECG waveform and the analysis results, and provide them to your doctor if necessary.

Conditions and Environments

The device cannot be used with a defibrillator.

The device cannot be used during CT or magnetic resonance imaging (MRI) test.

The device must not be used in a flammable environment (eg. oxygen-rich environment).

Do not swim or submerge the device in the water. Do not immerse the device in water or other liquids.

Do not place the device in the following environments: direct sunlight, high temperature, high humidity, close to water or fire sources, and high electromagnetic influence.

Do not place the device in a pressure vessel or gas sterilization equipment.

Try to avoid sweating. The sweat will degrade the contact between the electrodes and the skin, and thus affect the recording quality.

Inspect loosened electrodes, that can degrade performance or cause other problems.

Do not participate in violent or extensive physical activity in order to make appropriate measurements.

The device is not intended for use by infants weighing less than 10 kg.

The device should not be used on people with sensitive skin or allergies.

Battery

The built-in rechargeable lithium battery cannot be replaced. Non-professionals shall not open the casing, and modify or replace the battery.

Do not expose the main unit to high temperature such as ovens, water heaters and microwave ovens. Overheating may cause battery explosion.

If the battery leaks, keep your skin and eyes free from leaking liquids. If skin or eyes come into contact with leaking liquid, rinse your skin or eyes immediately and go to hospital for treatment.

Lithium batteries capacity will decrease after charge-discharge for 300 times.

Electrodes

If you purchase the disposable electrodes on your own, the electrode need have passed through a medical device registration.

Recommend to purchase the electrodes of the same model as the accompanied ones. Using others despite the recommendation may cause the analysis result inaccurate.

Check if the disposable ECG electrode has expired before use.

The disposable ECG electrodes shall not be applied to the traumatized or scarred skin.

Disposable ECG electrodes need be used in contact with the skin. If your skin is itching, irritating or ulcerated, stop using it immediately.

The electrodes should not contact other conductive parts including earth.

Cleaning

ER1 monitor and strap need to be cleaned regularly. Clean the device per week.

Carefully wipe the device with a clean, soft cloth or cotton ball with 75% medical alcohol or water.

Recycling

Do not casually dispose the main unit and its accessories at the end of their service life. They shall be disposed in accordance with local regulations. When they are intended to be discarded, they must be sent to the appropriate facility for recycling.

Troubleshooting

Cannot acquire data normally.

Solution:

Charge the device. If the problem continues, contact the distributor.

ECG waveform is of much interference.

Solution:

- Adjust the position of ER1 monitor.
- Replace the ECG disposable electrodes, or moisten the chest strap with water.

Recordings cannot be imported into the ECG Browser.

Solution:

- Charge the device.
- Try another operating system.

Appendix

Specifications

Classification		
EC Directive	MDD, 93/42/EEC	
	RED, 2014/53EU	
	ROHS 2.0, 2011/65/EU	
Degree protection against electrical shock	Type CF	
Environment		
	Operating	Storage
Temperature	5 – 45°C	-25 – 60°C
Relative humidity (non-condensing)	10% - 95%	10% - 95%
Atmospheric pressure	700-1060 hPa	700-1060 hPa
Degree of dust & water resistance	IP22	
Drop test	1.0 m	
Power supply		
Type of battery	Rechargeable lithium polymer battery	
Battery specification	3.7Vdc, 90mAh	
Battery run time	72 hours (fully charged)	
Charging input voltage range	4.5 - 5.5v DC voltage	

Charging time	2 hours (to 90% power)
ECG	
Lead type	Single-lead ECG
Lead	Lead II
Input impedance	$\geq 10\text{M}\Omega$, 10Hz
Linearity and dynamic range	10mV (peak-to-valley)
Common mode rejection	$\geq 60\text{dB}$
Frequency response	0.67 - 40 Hz
Gain error	Maximum error $\pm 10\%$
Physical	
Size	100×23×8.3 mm
weight	<35 g (with battery)
Expected service life	5 year

ECG Browser Installation Specifications

Hardware

CPU: P4 1.7GHz or above

Memory: 1GB or above

Storage: Free disk space greater than 200MB

Operating System

Support Windows 7 and 10

Symbols

Symbol	Description
	Manufacturer
	Date of manufacture
SN	Serial number
	Indicates a medical device that is not to be disposed of as unsorted municipal waste.
	Follow Instructions for Use.
	Type CF Applied Part
	MRI unsafe. Presents hazards in all MR environments as device contains strongly ferromagnetic materials.
IP22	Resistant to liquid ingress
	CE marking
	Authorized representative in the European community
	UKCA marking
	Authorized Representative in the United Kingdom
	This product complies with the rules and regulations of the Federal Communication Commission.

FCC Statement

FCC Warning:

FCC ID: 2ADXX-3613

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

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Shenzhen Viatom Technology Co., Ltd.
Prolinx GmbH
Prolinx Global LTD