

Care-Knee Support

Production description

When patient has knee pain, it is difficult to bend in any direction and affect the flexibility of the knee joint, so it is necessary to ensure that the TENS electrode is placed in the correct position. The knee support uses conductive silver fiber to stimulate the knee; this is an effective alternative. The use of knee support makes it easy for patients who lack this flexibility to position the knee joint.



Presentation

It is designed to place electrodes on knee pain sites easily. It is simple for patients to place it around the knee.

Use at Home, Work or on-the-Go

The wearable care-knee allows you to have a direct targeted knee therapy, so the patients can reduce pain and return to active lifestyle.


Instructions for use

1. Skin must always be clean, moist and free from lotion.
2. Insert one end of the cable to the metal snaps to connect the Care-Knee Support and the other end to the self-adhesive electrode pad. Make sure they are properly connected

for the good performance. Please refer to right figure.

3. Make sure the knee support are optimally positioned around the knee. Self-adhesive electrodes pad should be 40mm distance from Care-Knee Support..

4. Make sure that the TENS device is OFF. Connect cable plug to TENS device. After connection, turn on TENS device.

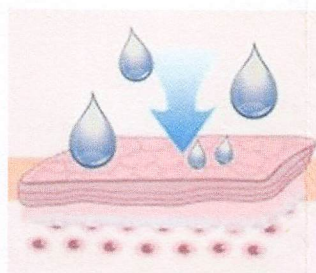

The area used needs to keep the skin moist



5. When exiting the application, turn off the TENS device at first. Remove the connector and Care-Knee Support.

 **Caution**

- In the course of the treatment, if feeling any discomfort or abnormal, stop immediately and contact a doctor
- Do not apply to broken skin. If skin burning occurs, stop use and contact your physician immediately.
- If the electrode pads are damaged, don't use.
- Adjust stimulator according to device and/or physician instructions for your treatment. For your safety and comfort, turn off the stimulator before you attempt to remove knee support from your body.
- The unattended use of children or incapacitated persons may be dangerous.
- If some people feel skin irritation from the conductive silver fiber on the knee support, stop using the knee support.

Cleaning and maintenance

To clean the product, wash it in warm water with a neutral detergent instead of hot water. Thoroughly remove all traces of the wash solution in warm water, do not wring. Wrap it in a terry cloth towel. Use a towel to remove excess water. Remove the towel and let it air dry at room temperature. Keep away from hot source and sunlight.



Shenzhen Roundwhale Technology Co., Ltd.

Add: 202, 2/F, Building 27, Dafa Industrial Park, Longxi Community, Longgang Street, Longgang District, Shenzhen, China.

Tel: +86-755-23212776

Fax: +86-755-23212776

E-mail: info@roovjoy.com

Website: www.roovjoy.com



Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestrasse 80, 20537 Hamburg, Germany

Email: shholding@hotmail.com